


**PHILIPS**  
RESPIRONICS

**SideStream**

**Nebulizer**

# Quick start guide

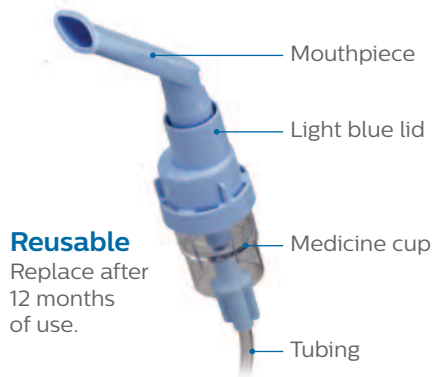


**Reusable**  
Replace after  
12 months  
of use.




**Disposable**  
Replace after  
4 weeks  
of use.

Before using SideStream for the first time, read the instructions for use booklet.


## How to use your SideStream nebulizer

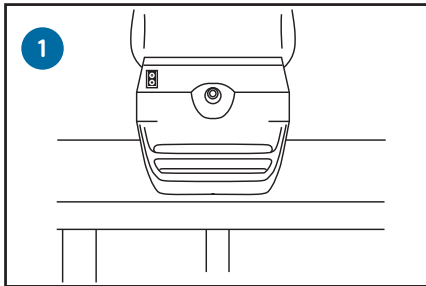


1 Place compressor on a solid surface.

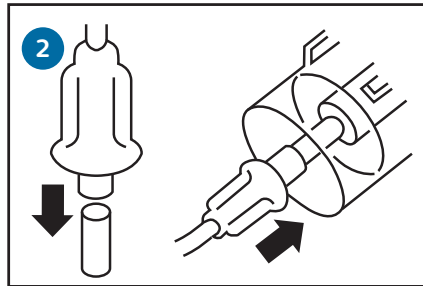


2 Connect tubing to the compressor and medicine cup.

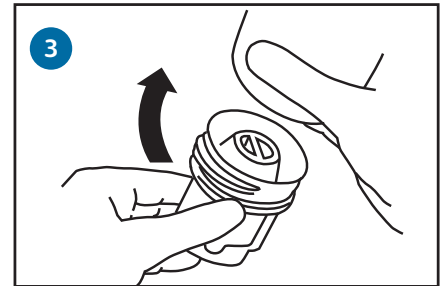


3 Unscrew lid.

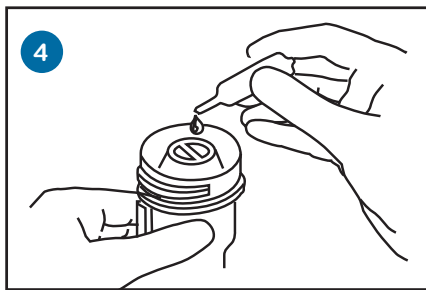


4 Fill medicine cup.

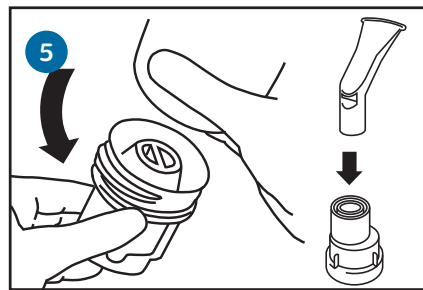


5 Replace lid. Connect mouthpiece.

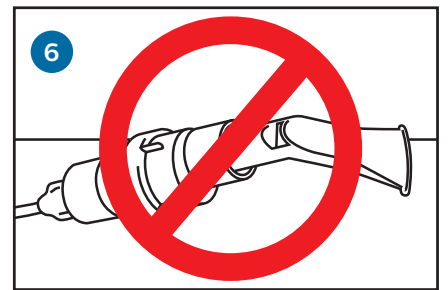


6 Do not tip SideStream.

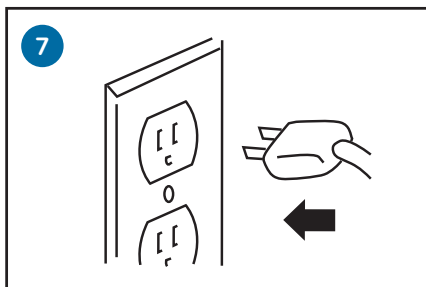

7 Plug in compressor.

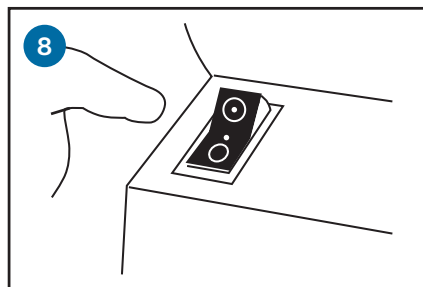


8 Turn switch on.

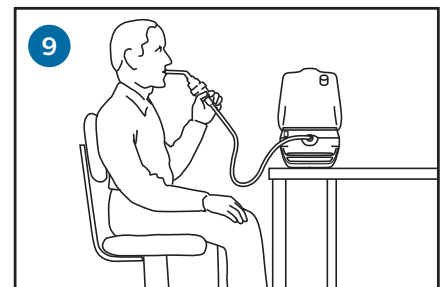


9 Sit upright, relax, and slowly breathe in and out.


## If you need to use a SideStream adult or pediatric mask...

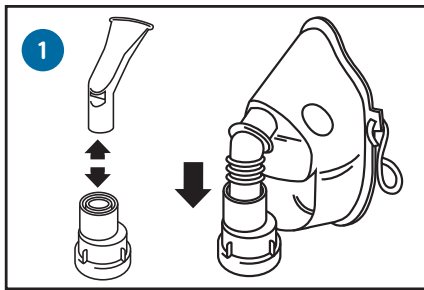

Replace mouthpiece with mask adaptor and mask.

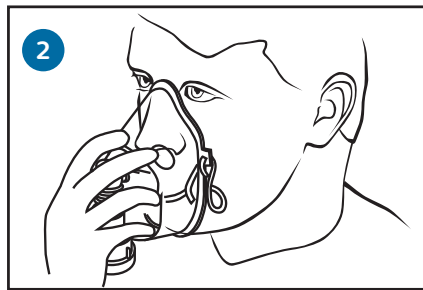


Place mask over nose and mouth.

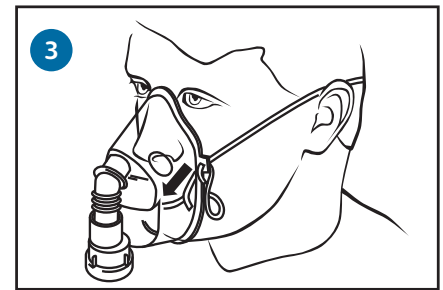


Place strap over head and adjust.

## Cleaning instructions


After every use (for both SideStream reusable and disposable)...

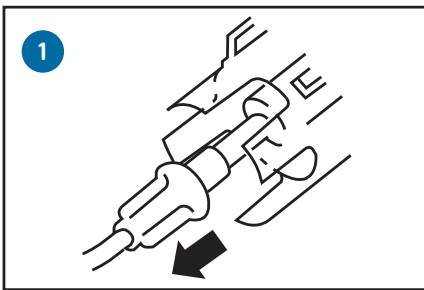

Disconnect handset from tubing.

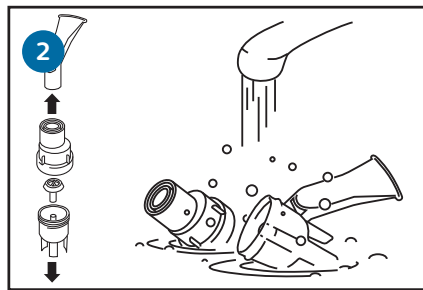


Disassemble and wash all parts in warm, soapy water (except the tubing).

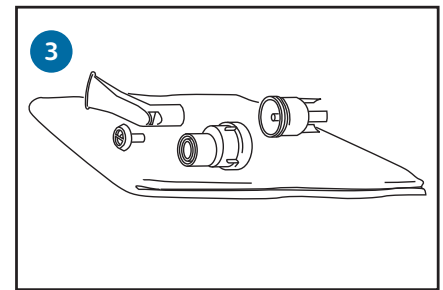


Rinse in clean water and air dry.


Once each week (for SideStream reusable only)...

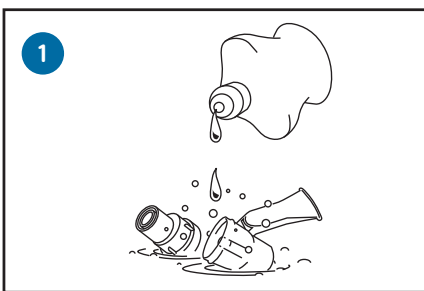

Place parts (except tubing) in water and 2-3 drops of dishwashing liquid.

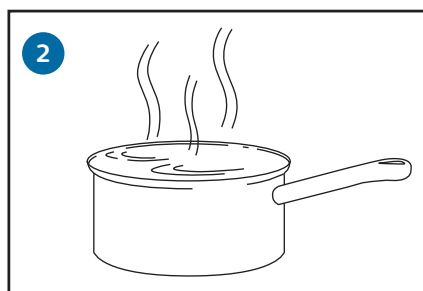

Boil 10 minutes.

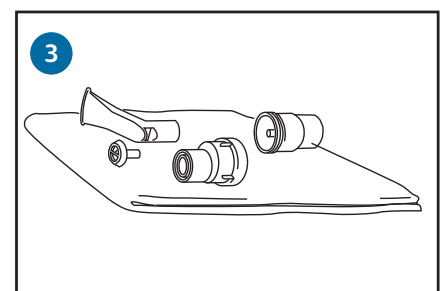

Rinse in clean water for 2 minutes and air dry.

©2016 Koninklijke Philips N.V.  
All rights reserved. Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.



Philips Respirics  
1010 Murry Ridge Lane  
Murrysville, PA 15668

[www.philips.com/respirics](http://www.philips.com/respirics)

Printer XX XX/XX/XX MCI xxxxxxxx PN xxxxxxx