


How to use your Sami nebulizer compressor


A nebulizer turns liquid medicine into an aerosol so you can breathe it deep into your lungs. Tucker uses his nebulizer when he is at home. Remember to have an adult help you use and clean your nebulizer.

Getting ready


- 1 Put your compressor on a table.

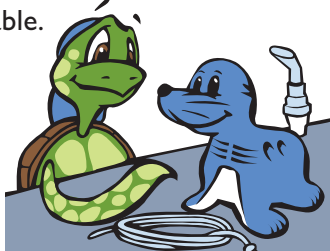

- 2 Plug the compressor into an electrical outlet.

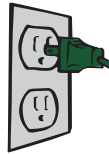

- 3 Connect one end of the tubing to the medicine cup and the other end to the compressor.

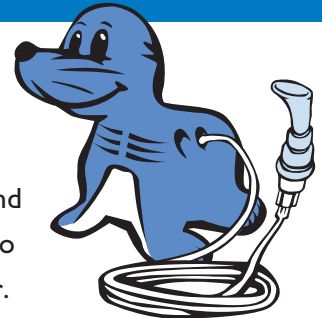


Taking your medication

- 4 Fill the medicine cup with your medication.




- 5 Turn the compressor on and check to see that a mist is coming out of the mouthpiece.

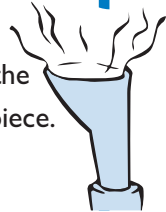

- 6 Sit up straight. Put the mouthpiece in your mouth and slowly breathe in and out. When a mask is used, you can breathe in and out the same way.




- 7 When the nebulizer begins to make a 'spluttering' noise, your treatment is finished. Turn off the compressor.

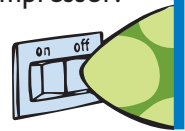


Cleaning your compressor

- 1 Unplug your compressor before cleaning.
- 2 Check the filter once a week and replace if dirty.

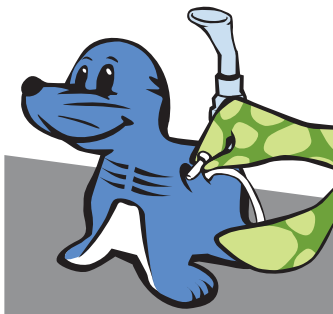

- 3 Wipe down the outside of the compressor once a month with a damp cloth.

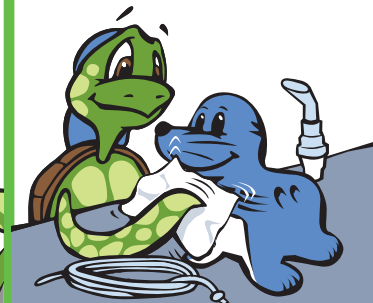

- 4 Store in a safe, dry place.

Cleaning your nebulizer

- 1 After every use disconnect the tubing and take apart the nebulizer.



- 2 Wash the nebulizer parts with warm soapy water.

- 3 Rinse with clean water and air dry.

- 4 Once a week, boil all parts (not the tubing or facemask) in water with 2-3 drops of dishwashing liquid for 10 minutes.



**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Asia

+49 7031 463 2254

Europe, Middle East, Africa

+49 7031 463 2254

Latin America

+55 11 2125 0744

North America

+1 425 487 7000

800 285 5585 (toll free, US only)

Customer Service

+1 724 387 4000

800 345 6443 (toll free, US only)

**Philips Respironics International
Headquarters**

+33 1 47 28 30 82

Philips Respironics Asia Pacific

+65 6882 5282

Philips Home Healthcare Solutions Australia

+61 (2) 9947 0440

1300 766 488 (toll free, Australia only)

Philips Respironics China

+86 021 24127311

Philips Respironics Deutschland

+49 8152 93 06 0

Philips Respironics France

+33 2 51 89 36 00

Philips Respironics Italy

+39 039 203 1

Philips Respironics Sweden

+46 8 120 45 900

Philips Respironics Switzerland

+41 6 27 45 17 50

Philips Respironics United Kingdom

+44 800 1300 845

Respironics New Jersey, Inc.

5 Wood Hollow Road
Parsippany, NJ 07054 USA

www.philips.com/respironics

Respironics, SideStream, Sami the Seal and Tucker the Turtle are trademarks of Koninklijke Philips Electronics N.V. and its affiliates. All rights reserved.



Please visit www.philips.com/asthma to find out more



© 2012 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

MCI 4104985 1101860 DD JUL 2012