


# How to use your peak flow meter


A peak flow meter tells you how well air moves out of your lungs. You should use it everyday because it can help you know if your asthma is getting worse before you might even feel it. Tucker uses his peak flow meter regularly as part of the asthma action plan he has worked out with his doctor. Remember to have an adult help you use and clean your peak flow meter.

## Getting ready

- 1 Move the indicator to the bottom of the scale.

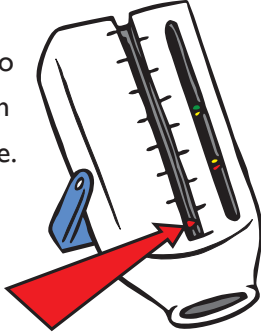

- 2 Stand up.

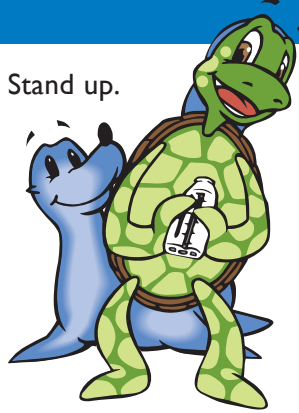


- 3 Take a big deep breath in, then...




## Blow out hard and fast

- 4 Put the peak flow meter in your mouth and close your lips around it (make sure your tongue isn't in the hole).

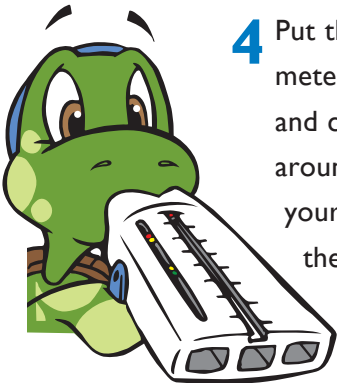


- 5 Blow out really hard and fast in one blow.

If you coughed or messed up don't worry, just do it again. Write down the number you get. Try two more times and write the biggest number of your three tries in your asthma diary.

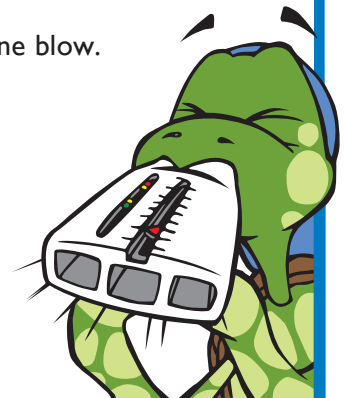

## Cleaning your peak flow meter


- 1 Make sure you clean your peak flow meter regularly.
- 2 Wash with warm soapy water or place on the top rack of your dishwasher.

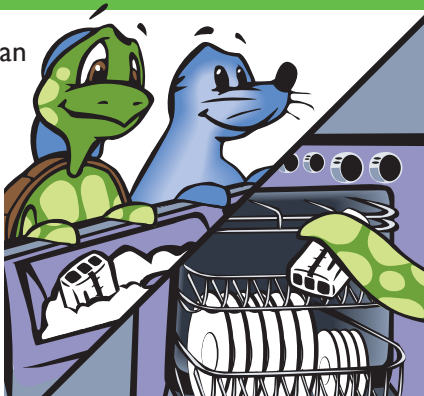

- 3 Rinse with clean water.



- 4 Allow to air dry.
- 5 Store in a safe, dry place.

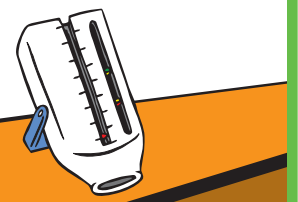

**Philips Healthcare is part of  
Royal Philips Electronics**

**How to reach us**

[www.philips.com/healthcare](http://www.philips.com/healthcare)  
[healthcare@philips.com](mailto:healthcare@philips.com)

Asia  
+49 7031 463 2254

Europe, Middle East, Africa  
+49 7031 463 2254

Latin America  
+55 11 2125 0744

North America  
+1 425 487 7000  
800 285 5585 (toll free, US only)

Customer Service  
+1 724 387 4000  
800 345 6443 (toll free, US only)

Philips Respironics International  
Headquarters  
+33 1 47 28 30 82

Philips Respironics Asia Pacific  
+65 6882 5282

Philips Home Healthcare Solutions Australia  
+61 (2) 9947 0440  
1300 766 488 (toll free, Australia only)

Philips Respironics China  
+86 021 24127311

Philips Respironics Deutschland  
+49 8152 93 06 0

Philips Respironics France  
+33 2 51 89 36 00

Philips Respironics Italy  
+39 039 203 1

Philips Respironics Sweden  
+46 8 120 45 900

Philips Respironics Switzerland  
+41 6 27 45 17 50

Philips Respironics United Kingdom  
+44 800 1300 845

Respironics New Jersey, Inc.  
5 Wood Hollow Road  
Parsippany, NJ 07054 USA

[www.philips.com/respironics](http://www.philips.com/respironics)

Respironics, Sami the Seal and Tucker the Turtle are trademarks of Koninklijke Philips Electronics N.V. and its affiliates. All rights reserved.



Please visit [www.philips.com/asthma](http://www.philips.com/asthma) to find out more



© 2012 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

MCI 4105087 1102898 DD AUG 2012