

GCX Mounting Assembly Installation Guide

Philips MP60/70 IntelliVue Keyboard Mount



The purpose of this guide is to describe attachment of Keyboard Mount to MP60/70 Mounting Adapter.

Parts Reference

The parts list below includes only the parts and hardware that will be used in this installation procedure (see photos for parts; hardware not shown).

Item #	Description	Qty
1	Keyboard Mount	1
2	M6 x 12mm Flat Head Machine Screw (FHMS)	4
3	Velcro Strip - Hook, 1" x 9"	2
4	Velcro Strip - Loop, 1" x 9"	2


Keyboard Mount



Tools Required: Phillips screwdriver.



Installation Note: This Keyboard Mount is compatible only with MP60/70 Mounting Adapters manufactured after 09/03 (see photo). The Keyboard Mount will not fit other mounting adapters.

Post-9/03 MP60/70 Mounting Adapter

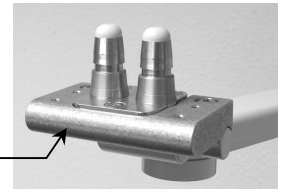


Attaching the Keyboard Mount

1. Attach Velcro Strips to the top side of the Keyboard Mount and the bottom of the Keyboard (suggested attachment shown below). **Installation Note:** Hook side of Velcro must be aligned opposite loop side.

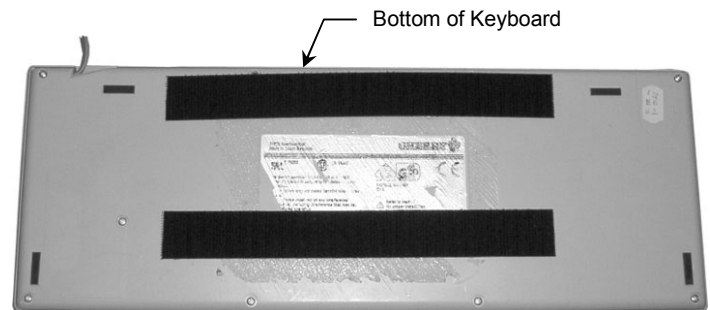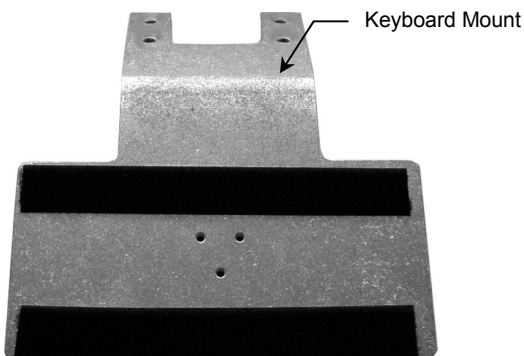

2. Attach the Keyboard Mount to the bottom of the MP60/70 Mounting Adapter using four (4) M6 x 12mm FHMS (see photos below). **Installation Note:** The photos below show a Keyboard Mount attachment to a Mounting Adapter on an M-Series Arm. Keyboard Mount attachment is identical for all other Mounting Systems with an appropriate MP60/70 Mounting Adapter (post-9/03).

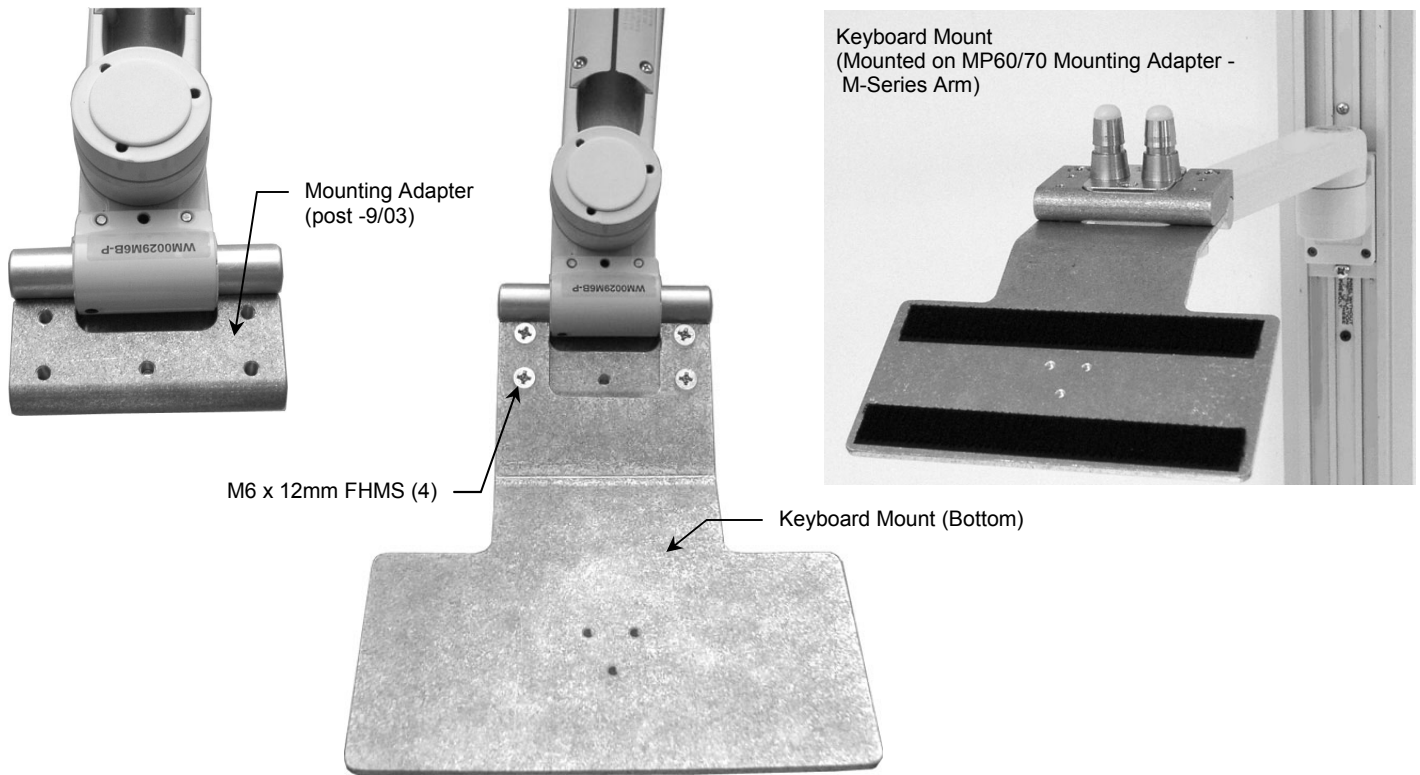

3. Press the keyboard firmly onto the Keyboard Mount.