

PHILIPS

InnoSpire Go

Portable
mesh nebulizer



Reduces treatment time by 25%*

The easy-to-use and virtually silent InnoSpire Go has been specifically designed to shorten nebulizer treatment times, giving your patients more time for life.

Utilizing established, clinically proven vibrating mesh technology adopted by hospitals worldwide, InnoSpire Go delivers all commonly prescribed respiratory medications.



As easy as 1, 2, 3...



1. Fill

Lift the green lid, pour in the medication and the InnoSpire Go is ready to use.



2. Treat

Press the on button and the InnoSpire Go will begin to nebulize.



3. Clean

Remove the mouthpiece and quickly rinse it in warm water.

* Versus predecessor Aeroneb Go.

InnoSpire Go provides an easy therapy experience, which helps patients administer therapy correctly.



Treatment in 4 mins**

Efficient aerosol therapy with audible and visible indicators that signal therapy completion - before automatically turning off.



Virtually silent

A small, portable and discreet therapy device, allowing your patients to take their therapy when and where they need it.



Built-in battery

Rechargeable, long life, built-in battery that provides up to 30 treatments between each charge.



Simple, only two parts

The easily detachable mouthpiece makes the InnoSpire Go simple to use, clean and maintain, encouraging greater adherence.***

InnoSpire Go aerosol data

	Salbutamol 2.5 ml	Ipratropium bromide 2 ml	Budesonide 2 ml
MMAD µm	3.99	3.93	4.84
Respirable fraction % particles < 5 µm	64.4%	65.3%	52.1%
Treatment time (min:sec)	4.18	3.44	3.34
Residual volume ml	0.3	0.2	0.3



Product specification table

Battery	Lithium ion
Dimensions	7.0 cm x 4.5 cm x 13.5 cm
Weight	111 g
Warranty	Handset 2 years Mouthpiece 1 year

** Using 2.5ml salbutamol

*** A mask is included for children that find it difficult to use the mouthpiece.

Find out more – www.philips.com/respironics

