Noninvasive Blood Pressure Cuffs

Single Care

REF

989803182281
989803182291
989803182301
989803182311
989803182321

Instructions for Use

CE Gentle Care

REF

M4572B, M4573B M4574B,
M4575B, M4576B M4577B,
M4578B

Multi Care

REF

989803183311
989803183321
989803183331
989803183341
989803183351
989803183361
989803183371

CE Easy Care

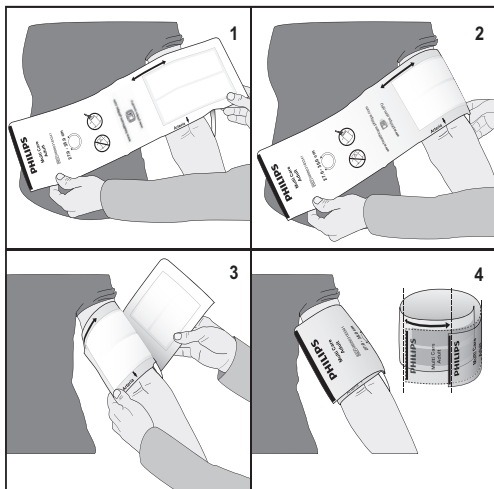
REF

M4552B, M4553B M4554B,
M4555B M4556B, M4557B
M4558B, M4559B

CE Comfort Care

REF

M1571A, M1572A, M1573A,
M1574A, M1574XL,
M1575A, M1576A

PHILIPS

Contents

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EN - Instructions for Use

Associated Documents

- *NBP Cuffs and Air Hoses Addendum - Care, Cleaning, and Disinfection*
- *Air Hoses for NBP Cuffs Instructions for Use*

Intended Use

Philips multi-patient and single-patient cuffs are to be used with compatible devices intended for use by, or under the supervision of, a licensed physician or other healthcare provider for the noninvasive measurement of infant, pediatric, and adult human blood pressure.

Indications

Philips multi-patient and single-patient cuffs are indicated for the measurement of noninvasive blood pressure and pulse rate for monitoring purposes. These devices are limited by the indications for use of the connected monitor in healthcare facilities. These devices are intended to interact with patient intact skin only.

Contraindications

There are no known contraindications.

Compatibility

NBP Cuffs and Air Hoses can be used with any monitor or defibrillator that lists the cuffs and air hoses as accessories in that product's Instructions for Use.

WARNINGS

- Turn off auto function or remove cuff from monitor when not on patient. This may lead to cuff leaks.
- When cleaning cuffs, use care to prevent fluid from entering tubing. Use end cap, if provided with cuff. Do not immerse tubing in any fluids. This may lead to cuff or monitor failure.
- Avoid placing cuff on patient in a manner that can lead to a cuff tube becoming kinked (a kinked tube can cause inaccurate readings).
- Single-patient (disposable) cuffs are not intended for use on more than one patient.

Cautions

Federal (US) law restricts this device to sale by or on the order of a medical practitioner.

Visual Inspection Prior to Use

Before using a cuff on a patient, perform a visual inspection to determine if the cuff has reached end-of-life. To determine if the product has reached its end of life, visually inspect the cuff for the following damage or deterioration conditions:

- hook and loop not fastening
- cuff leaks
- damaged or frayed material

Whenever visual inspection reveals a cuff is no longer suitable for continued use or does not perform as intended, dispose of the cuff. Refer to “Disposal” on page 3.

Applying the Noninvasive Blood Pressure Cuff

Note: Refer to the front cover. Numbers in bold in the following procedure correspond to the figure numbers on the front cover.

Always inspect the device to ensure it is suitable for use.

- 1 Select the proper cuff size for the patient’s limb circumference. Improper cuff size selection may lead to inaccurate readings. See “Select Appropriate Cuff Size” on page 3.
- 2 Apply cuff to limb (**1**) such that the side displaying the Caution symbol () is toward the patient and the arrow labeled **Arteria** is over the limb artery.
- 3 Wrap the cuff snugly around the patient’s limb (**2** and **3**). Ensure that the index edge falls within the range indicated by the range arrow (**4**) and arteria arrow remains correctly located over the limb artery.
- 4 Secure the connector on the cuff to the air hose connector to prevent leaking. To select the proper hose for use with the noninvasive blood pressure cuff, refer to *Air Hoses for NBP Cuffs Instructions for Use*.

Select Appropriate Cuff Size

Philips noninvasive blood pressure (NBP) cuffs are available to suit a variety of patient applications and are color-coded for size selection.

Cuff Size (Color)	Limb Circumference (cm)				
	SINGLE CARE	Gentle Care	MULTI CARE	Easy Care	Comfort Care
Infant (Orange)		10.0–15.0	10.0–15.0	10.0–15.0	10.0–15.0
Pediatric (Green)	14.0–21.5	14.0–21.5	14.0–21.5	14.0–21.5	14.0–21.5
Small Adult (Blue)	20.5–28.5	20.5–28.5	20.5–28.0	20.5–28.5	20.5–28.0
Adult (Navy)	27.5–36.0	27.5–36.0	27.0–35.0	27.5–36.0	27.0–35.0
Adult Long ¹ (Navy)	27.5–36.0	27.5–36.0	27.0–35.0	27.5–36.0	27.0–35.0
Large Adult (Burgundy)	35.0–45.0	35.0–45.0	34.0–43.0	35.0–45.0	34.0–43.0
Large Adult Long ¹ (Burgundy)		35.0–45.0		35.0–45.0	
Thigh (Grey)		44.0–56.0	42.0–54.0	44.0–56.0	42.0–54.0

¹ Edge striping denotes a Long cuff.

Extra-Long (XL or Long) cuffs are for use only when the standard cuff size cannot be used on a patient due to fit.

Caution: With the use of Extra-Long cuffs outside the recommended range, there may be limitations to the accuracy of the measurement, as closure of any cuff on a circumference larger than the cuff's recommended range can result in incorrectly high readings.

Cleaning and Disinfection

Refer to *NBP Cuffs Care and Air Hoses Addendum - Care, Cleaning, and Disinfection* for a list of validated cleaners and disinfectants and the appropriate procedures.

Disposal

Dispose of any cuff that becomes damaged or deteriorated. Follow approved disposal methods for medical waste as specified by your facility (hospital, clinic) or local regulations.

Reordering Information

The following products are CE marked and available in the European Union.

Easy Care:

Part	Description	Part	Description
M4552B	Easy Care Cuff, Infant	M4556B	Easy Care Cuff, Adult XL
M4553B	Easy Care Cuff, Pediatric	M4557B	Easy Care Cuff, Large Adult
M4554B	Easy Care Cuff, Small Adult	M4558B	Easy Care Cuff, Large Adult XL
M4555B	Easy Care Cuff, Adult	M4559B	Easy Care Cuff, Thigh

Gentle Care:

Part	Description	Part	Description
M4572B	Gentle Care Cuff, Infant	M4576B	Gentle Care Cuff, Adult XL
M4573B	Gentle Care Cuff, Pediatric	M4577B	Gentle Care Cuff, Large Adult
M4574B	Gentle Care Cuff, Small Adult	M4578B	Gentle Care Cuff, Large Adult XL
M4575B	Gentle Care Cuff, Adult		

Comfort Care:

Part	Description	Part	Description
M1571A	Comfort Care Cuff, Infant	M1575A	Comfort Care Cuff, Large Adult
M1572A	Comfort Care Cuff, Pediatric	M1574XL	Comfort Care Cuff, Adult XL
M1573A	Comfort Care Cuff, Small Adult	M1576A	Comfort Care Cuff, Thigh
M1574A	Comfort Care Cuff, Adult		

Incident Reporting

Any serious incident that has occurred in relation to this device should be reported to Philips and the competent authority of the European Economic Area (EEA) countries, including Switzerland and Turkey in which the user and/or patient is established.

Environmental Specifications

Environment	Specification
Storage	-20 °C to +60 °C (-4 °F to +140 °F); 0 to 90 % Relative Humidity at 60 °C (140 °F)
Operating (Comfort Care, Single Care, Multi Care, Easy Care)	0 °C to + 50 °C (32 °F to 122 °F) at 90 % Relative Humidity
Operating (Gentle Care)	0 °C to +40 °C (32 °F and 104 °F) at 90 % Relative Humidity

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Gentle Care:

Osa	Kirjeldus	Osa	Kirjeldus
M4572B	Gentle Care Cuff, Infant	M4576B	Gentle Care Cuff, Adult XL
M4573B	Gentle Care Cuff, Pediatric	M4577B	Gentle Care Cuff, Large Adult
M4574B	Gentle Care Cuff, Small Adult	M4578B	Gentle Care Cuff, Large Adult XL
M4575B	Gentle Care Cuff, Adult		

Comfort Care:

Osa	Kirjeldus	Osa	Kirjeldus
M1571A	Comfort Care Cuff, Infant	M1575A	Comfort Care Cuff, Large Adult
M1572A	Comfort Care Cuff, Pediatric	M1574XL	Comfort Care Cuff, Adult XL
M1573A	Comfort Care Cuff, Small Adult	M1576A	Comfort Care Cuff, Thigh
M1574A	Comfort Care Cuff, Adult		

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Copyright © 2020
Nederland B.V.