

Scanning patients with MR Conditional implants

Download this FieldStrength article

FieldStrength MRI articles

Educational - December 2015

Clearly specified conditions are key

With an aging population, large joint replacements and cardiac implantable devices are becoming increasingly prevalent. Also the prevalence of conditions needing an MRI examination, such as neurodegenerative diseases, cancer, and musculoskeletal diseases, increases with age.

“12.5 million people in the USA are presently carrying an orthopedic or cardiac implant.”

Not all implanted devices are an absolute contraindication for MRI anymore. Patients with MR Conditional implants can undergo MRI, but only under clearly defined conditions and performed by well-trained MRI staff. In this article, four MRI experts discuss the scanning of patients with MR Conditional implants. But first of all, how relevant is this topic?



Emanuel Kanal, MD, FACR, FISMRM, AANG

Professor of Radiology and Neuroradiology and Director of Magnetic Resonance Services at the University of Pittsburgh, Pittsburgh, USA. He is Chairman of the American Board of Magnetic Resonance Safety and is lead author of the American College of Radiology's White Paper on MR Safety and MR Safe Practice Guidelines.



Greg Brown

MRI Consultant and PhD candidate at the Centre for Advanced Imaging, The University of Queensland, St Lucia, Australia. He is an Honorary member of the SMRT, and serves on their MR safety group. He is a non-voting Board Member and SMRT delegate on the American Board of Magnetic Resonance Safety, and holds their MRSO certification. Mr. Brown is an active user of social media platforms on the use of MRI and MRI safety.



Paul W. de Bruin, PhD

Medical Physicist at the Radiology Department, Leiden University Medical Center, Leiden, The Netherlands. As a member of the Clinical Physics Group and the C.J. Gorter Center for High-field MRI at Leiden University he has various research interests in the field of MRI including high-field MR (3T/7T) applications, DCE-MRI, sodium MRI, pharmacokinetics, and musculoskeletal applications.



Harald Kugel, PhD

MR Physicist at the Department of Clinical Radiology of the University of Münster, Münster, Germany. He is the author of numerous articles on MR applications and safety, has been actively involved in the teaching of practical ISMRM courses in MR safety, and is a member of the Technical Committee 'MR Procedures' of the German Standards Committee Radiology.

Metallic and electronic implants are on the rise

In five major European countries (France, Germany, Italy, Spain, and the UK) more than 2.5 million large joint reconstructions and spinal implant procedures were performed in 2015, a figure expected to rise to more than 3 million in 2020 [1].

In those same countries, the prevalence of such passive orthopedic implants was 19% among people over the age of 65 years, which is predicted to rise to 30.5% in 2020 [1]. The prevalence of active cardiac implants is also expected to rise from the current 10.4% in the over 65 population to 11.8% in 2020.


	2015	2020
# Procedures	> 3 M	> 3.4 M
Total population	321 M	326 M
% with implant	3.9%	7.2%
Total 65+	63 M	68 M
% 65+ with implant	13.3%	28%

A similar situation is seen in the USA where 17% of the over 65 population are estimated to carry a large joint or spinal implant and 10.4% have a cardiac implant, figures predicted to rise to 26% and 11.8%, respectively, by 2020 [1].

	2015	2020
# Procedures	> 2.6 M	> 3.1 M
Total population	322 M	345 M
% with implant	3.1%	5.6%
Total 65+	48 M	56 M
% 65+ with implant	21.6%	31.3%


Orthopedic and cardiac implants in Europe

Forecast of number of procedures per year [1]

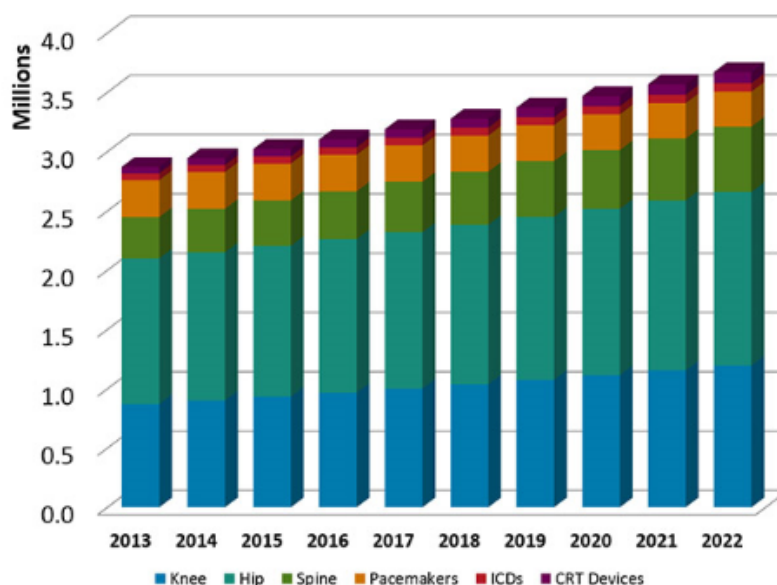

Orthopedic and cardiac implants in USA

Forecast of number of procedures per year [1]

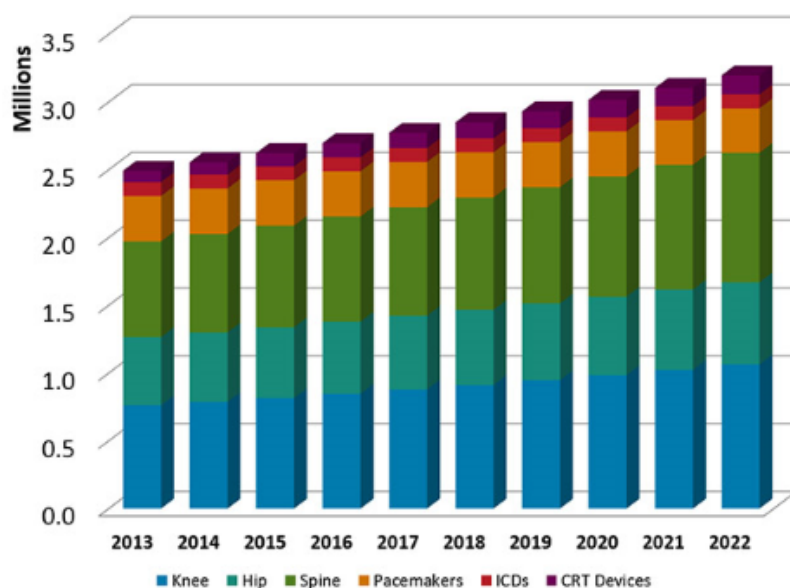


Patients with implants often need MRI scans

There is a significant clinical need for patients with orthopedic implants to undergo MRI examinations. Besides the increased prevalence of medical implants in older people, the likelihood that the average person will need an MRI scan during the next 10 years increases with age from around 47% at the age of 30, to around 69% at the age of 70 [1].

Combining these numbers with the data on implant incidence suggests that, for instance, in the USA, 5.7 million patients over 65 and carrying orthopedic or cardiac implants will need an MRI scan within 10 years.

And this will rapidly rise to about 12.6 million patients in 2020, a doubling of the number in five years.

A recent USA-based study of patients with spinal cord stimulation (SCS) implants estimated that about 82–84% of SCS-implanted patients will need a spinal MRI scan within 5 years of receiving their SCS implant. A further 59–74% of patients will need a non-spinal MRI scan within 10 years [2].

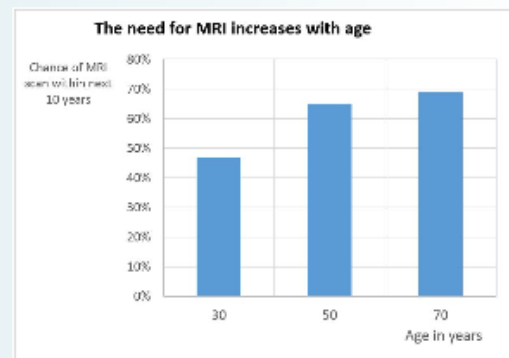

“There are only few implants for which scanning is an absolute contraindication.”

Which patients with implants can be scanned?

There are currently above 34 million MRI patient exams per year within the USA [3]. About 3.9% of the US population – 12.5 million people of whom 10.3 million are over 65 – are presently carrying an orthopedic or cardiac implant. Due to safety concerns, patients carrying some kind of metallic medical implant or device may potentially be denied an MRI scan. But are such concerns justified?

“There are only few implants for which scanning is an absolute contraindication, but we can scan patients with an MR Safe or MR Conditional implant” says Emanuel Kanal, MD, Director of Magnetic Resonance Services and Professor of Radiology and Neuroradiology at the University of Pittsburgh Medical Center, USA. As Dr. Kanal explains, many patients who are referred for an MRI scan have an implant of some kind. “At an academic center the chances of a patient having an implant are much higher than at free standing, private practice environment. I would guesstimate that at our site the number of patients with implants is somewhere between 10% and 25%.”

According to Greg Brown, MRI technologist studying at the Centre for Advanced Imaging, University of Queensland, Australia, a similar situation exists in his country.

Understanding implant types and scanning conditions: MR Safe, MR Conditional, MR Unsafe

“There may be certain devices or implants that at certain levels of radiofrequency power may be potentially dangerous to scan,” says Dr. Kanal. Such implants may interfere with the MRI-related RF fields inside the body resulting in increased risks to the patient due to local hot spots.

“An MR safe implant has no potential interaction with a scanner,” says Mr. Brown. “So that would be non-conducting, non-magnetic objects. But other implants have the label ‘MR Conditional’ and that term is really quite important.”

All medical implants have to be tested by their manufacturers for MR safety and labeled according to standardized terminology [4].

Active implants are those that contain a power source (such as cardiac pacemakers and spinal cord stimulators), whereas passive implants have no power source (such as aneurysm clips and replacement joints).

Definitions and icons*



MR Safe

An item that poses no known hazards resulting from exposure to any MR environment. MR Safe items are composed of materials that are electrically nonconductive, nonmetallic, and nonmagnetic.



MR Conditional

An item with demonstrated safety in the MR environment within defined conditions. At a minimum, address the conditions of the static magnetic field, the switched gradient magnetic field and the radiofrequency fields. Additional conditions, including specific configurations of the item, may be required.



MR Unsafe

An item which poses unacceptable risks to the patient, medical staff or other persons within the MR environment.

*Reprinted, with permission, from ASTM F2503-13 *Standard Practice for Marking Medical Devices and Other Items for Safety in the Magnetic Resonance Environment*, copyright ASTM International, 100 Barr Harbor Drive, West Conshohocken, PA 19428. A copy of the complete standard may be obtained from ASTM, www.astm.org.

Are patients unnecessarily denied scans?

Although comprehensive guidelines for the safe use of MRI have been issued by professional bodies such as the American College of Radiology (ACR) [5] and Medicines and Healthcare Products Regulatory Agency [6], some confusion remains in everyday practice. "As a result, patients with implants who need an MRI scan may be either never referred or are denied the scan, this is very much dependent on the site," Mr. Brown explains.

"Patients with implants are denied MRI scans by some places. But as a major hospital site, we might be looking at just a couple of patients that we really can't scan, maybe 1-2%. At a site where they are not as comfortable with the safety aspects or don't want to spend the time on it, they might be rejecting more."

"The lack of awareness by referring physicians and even radiology experts can be problematic," says Dr. Kanal. "Many times people tell us they didn't even bother sending their patients with conditional implants for an MRI. Instead, they sent them straight for a CT, thinking they could not get safely scanned on MRI."

"Blanket rules, like not scanning pacemakers, are not serving the patients very well."

Radiology should lead in changing perceptions

"I think there is a lack of understanding among radiologists and technologists, let alone referring physicians," says Dr. Kanal. "But I don't believe it is the referring physicians' responsibility. I think it is the responsibility of radiology to educate them and to explain that we can perform an MRI on a certain patient by scanning under certain specific conditions."

"The perception that all active implants, such as pacemakers cannot be scanned, is also incorrect," says Mr. Brown. "This may still refer back to old guidance coming from early practice. The British Heart Rhythm Society has just released guidelines for scanning MR Conditional pacemakers [7]. And there has been a 2015 German publication on the same topic [8]. Both are trying to say that we need to think more about this, because the patients are going to need these scans. So blanket rules, like not scanning pacemakers, are not serving the patients very well."

"Scanning patients with MR Conditional implants inevitably brings its own challenges."

Finding the conditions before scanning

"Scanning patients with MR Conditional implants inevitably brings its own challenges," explains Paul W. de Bruin, PhD, medical physicist in the Radiology Department at Leiden University Medical Center, the Netherlands. "It involves two steps that cost us time. The first one is figuring out exactly which implant the patient has. That could be improved by having some sort of registry to look up which implant is in the patient. The second is that it can sometimes be difficult to find the information from the implant manufacturer. For instance, when we find that a certain implant is labeled MR Conditional, it means that we can scan under certain conditions. We then have to figure out what those conditions are by going to the implant manufacturer's website. But it's often not straightforward to find the information. After that we need to pay special attention when we bring the patient to the scanner and when setting the sequence parameters at the scanner to remain within the condition limits throughout the exam."

These suggestions are reinforced by Mr. Brown. "The means to rapidly identify the exact device in a patient would help MR sites. A central internet-accessible repository by manufacturers of the instructions for use and MR conditions available for all their devices, including ones no longer sold, would speed things up a lot for sites."

Setting up MRI scans for patients with MR Conditional implants

"Scanning patients with implants means having to be aware of the possible hazards," says Harald Kugel, PhD, MR physicist at the Institute of Clinical Radiology, University of Münster, Münster, Germany. "There are specific limits on specific parameters. One of the most common limits is on specific absorption rate (SAR) [9]. In general, this means that spin echo and TSE sequences should be avoided by switching to gradient echo instead. This is just an example of what needs to be done or considered."

"We have specific standard operating procedures, telling our staff what to do with specific implants. For common implants everybody in our team knows what to do. For instance, if a patient with a pacemaker comes for an MRI exam, we have a specific routine to check which type of pacemaker it is, to check that a cardiologist is present as an MR Conditional pacemaker usually needs to be switched in an MR-compatible mode, etc. So some actions have to be taken and this is all laid down in our procedure."

"Scanning patients with implants means having to be aware of the possible hazards."

Education is an important step

"I think a lot more education of technologists and radiologists is needed, so that we can develop an efficient and structured way forward," Mr. Brown says.

"We know that MRI scanners have to pass certain levels of safety and show that they are documented to be kept at certain guidelines and thresholds [10]. MR site accreditation in the USA [11] documents that the site is appropriately designed," says Dr. Kanal. "But who is missing in all this? There was no certification process to show that the magnetic resonance medical director, radiologist, technologist, or physicist have a comprehensive understanding of the safety issues associated with magnetic resonance environments or how to apply them. We therefore created the American Board of MR Safety in 2014. Its sole purpose is to certify and credential MR medical directors, MR safety officers, and MR safety experts, who represent the radiologists/physicians, technologists, and physicists who are charged with overseeing safety in clinical and research magnetic resonance environments."

Saying 'no' is easy, but saying 'yes' requires knowledge

"Sometimes patients have certain implants and the site is not sufficiently familiar with what can and can't be done to decrease the risk of an MRI scan," says Dr. Kanal. "They may choose, for ostensible 'safety' objectives, to not scan that patient. I put the word safety in quotes because not scanning a patient for whom a diagnostic MRI was requested has its own risks. The patient may go undiagnosed or may have to be sent for a more invasive study to make a diagnosis."

"Saying 'no' is easy, but saying 'yes' requires knowledge, confidence in that knowledge, and the willingness to say yes and to apply that knowledge."

"Not scanning a patient for whom a diagnostic MRI was requested can also potentially impact patient care."

References

1. Philips, data on file. Based on Millennium research group reports RPUS21LJ09; RPUS21LJ10; RPUS21LJ15; RPEU21LJ08; RPEU21LJ15; RPUS50DI14; RPUS20SP10; RPEU20SP10; RPUS21LJ10; RPEU20SP13; RPUS20SP15; US12CR07; RPGL12CR10; RPGL12CR14 and Barmer GEK Arztreport 2011.
2. Desai MJ, Hargens LM, Breitenfeldt MD, Doth AH, Ryan MP, Gunnarsson C, Safriel Y. *The rate of magnetic resonance imaging in patients with spinal cord stimulation.* Spine (Phila Pa 1976). 2015;40:E531-7.
3. IMV 2013 MR Market Outlook Report.
4. ASTM F2503-13, *Standard Practice for Marking Medical Devices and Other Items for Safety in the Magnetic Resonance Environment.* ASTM International, West Conshohocken, PA, 2013. www.astm.org*
5. Expert Panel on MR Safety. *ACR Guidance Document on MR Safe Practices: 2013.* J. Magn. Reson. Imaging 2013;37:501-30.
6. Medicines and Healthcare Products Regulatory Agency. *Safety Guidelines for Magnetic Resonance Imaging Equipment in Clinical Use.* 2014

7. Lowe MD, Plummer CJ, Manisty CH3, Linker NJ. *Safe use of MRI in people with cardiac implantable electronic devices.* Heart. 2015. pii: heartjnl-2015-308495. doi: 10.1136/heartjnl-2015-308495. [Epub ahead of print]
8. Sommer T, et al. *German Roentgen Society Statement on MR Imaging of Patients with Cardiac Pacemakers.* Fortschr Röntgenstr 2015; 187: 777-87.
9. Woods, TO. *Establishing Safety and Compatibility of Passive Implants in the Magnetic Resonance (MR) Environment,* 2014, www.fda.gov*
10. For instance: IEC 60601-2-33 – Requirements for the Safety of MR Equipment for Medical Diagnosis; FDA – Guidelines for Premarket Notifications for MR Diagnostic Devices; NEMA MS 1 through 9 – Safety and Performance Standards
11. American College of Radiology. *Accreditation.* www.acr.org/quality-safety/accreditation*

* When clicking this link you are leaving our website to access external pages over which Philips has no control and does not endorse.

Subscribe to **FieldStrength**

Our periodic FieldStrength MRI newsletter provides you articles on latest trends and insights, MRI best practices, clinical cases, application tips and more. Subscribe now to receive our free FieldStrength MRI newsletter via e-mail.



Stay in touch with Philips MRI

[Subscribe now](#)

Feedback



Related information

- [Metal artifact reduction for MRI of metal prostheses and implants](#) ›



More from FieldStrength

- [Overcoming motion challenges for first-time-right MR imaging – Radiologie am St. Joseph Stift, Germany](#) ›
- [Ingenia 1.5T S system for first-time-right MR imaging](#) ›



Connect with Philips



Country ›

[Cookie Notice](#) | [Privacy Notice](#) | [Investor Relations](#) | [Careers](#) | [Contact Philips](#) | [Philips](#) | [Privacy Policy](#) | [Terms of use](#) | [Site map](#)

© Koninklijke Philips N.V., 2004 – 2015. All rights reserved.