Manufactured for:

Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

Respironics Deutschland
Gewerbestrasse 17
82211 Herrsching, Germany
EC REP

©2012 Koninklijke Philips Electronics N.V. All rights reserved.

1077885 R04
CV 2/2/2012

Amara SE | The Philips Respironics logo is located at the bottom right. It features the word "PHILIPS" in blue above a curved swoosh line, and "RESPIRONICS" in a bold, sans-serif font below it. To the left of the swoosh is a small blue circular icon containing a stylized human figure reading a book.

Amara SE

EN	001	INSTRUCTIONS FOR USE	HU	106	HASZNÁLATI UTASÍTÁS
FR	008	MODE D'EMPLOI	SK	113	NÁVOD NA POUŽITIE
IT	015	ISTRUZIONI PER L'USO	SL	120	NAVODILA ZA UPORABO
DE	022	GEBRAUCHSANWEISUNG	LV	127	LIETOŠANAS INSTRUKCIJA
ES	029	INSTRUCCIONES DE USO	LT	134	NAUDOJIMO INSTRUKCIJA
PT	036	INSTRUÇÕES DE UTILIZAÇÃO	ET	141	KASUTUSJUHISED
PT-BR	043	INSTRUÇÕES DE USO	RO	148	INSTRUCȚIUNI DE UTILIZARE
EL	050	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	BG	155	ИНСТРУКЦИИ ЗА УПОТРЕБА
DA	057	BRUGSVEJLEDNING	HR	162	UPUTE ZA UPORABU
NL	064	GEBRUIKSAANWIJZING	ZH-CN	169	使用说明
SV	071	BRUKSANVISNING	ZH-TW	176	使用說明
FI	078	KÄYTTÖOHJEET	RU	183	ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
NO	085	BRUKSANVISNING	SR	190	UPUTSTVO ZA UPOTREBU
PL	092	INSTRUKCJA UŻYCIA	TR	197	KULLANMA TALİMATI
CS	099	NÁVOD K POUŽITÍ	KO	204	사용 설명서

Instructions for Use

Amara SE Full Face Mask

ENGLISH

Intended Use

The Amara SE Full Face Mask is intended to provide a patient interface for application of noninvasive ventilation. The mask is to be used as an accessory to ventilators which have adequate alarms and safety systems for ventilation failure, and which are intended to administer CPAP or positive pressure ventilation for treatment of respiratory failure, respiratory insufficiency or obstructive sleep apnea. The mask is for multi-patient use in the hospital/institutional environment only. The mask is to be used on patients (>66 lbs/30 kg) who are appropriate candidates for noninvasive ventilation

Note: An exhalation port is not built into the mask. A separate exhalation device must be used with this mask.

Note: This mask does not contain natural rubber latex or DEHP.

Symbols

Warning
or
Caution

Note

Tip

Does Not
Contain
Natural
Rubber Latex

Warnings:

- This mask is not suitable for providing life support ventilation.
- Hand wash prior to use. Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.
- This mask requires a separate exhalation device.
- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. Do not block or try to seal the exhalation port. Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of CPAP systems.
- This mask does not include an anti-asphyxia valve that would allow a patient to breathe if the ventilator failed.
- If oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. Explanation of the Warning: When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP and bi-level machines.
- At low CPAP or EPAP pressures the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.

- Consult a physician if any of the following symptoms occur: Unusual chest discomfort, shortness of breath, stomach distension, belching, severe headache, blurred vision, drying of the eyes, eye pain or eye infections.
- This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask.
- This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- Consult a physician or dentist if you encounter tooth, gum, or jaw soreness. Use of a mask may aggravate an existing dental condition.
- Attaching an exhalation device requires therapy pressure level adjustment to compensate for increased leak.
- A minimum of 3cm H₂O (hPa) must be maintained when using this mask.
- The mask contains small parts which could result in a choking hazard.
- Remove the O₂ enrichment attachment when not in use.

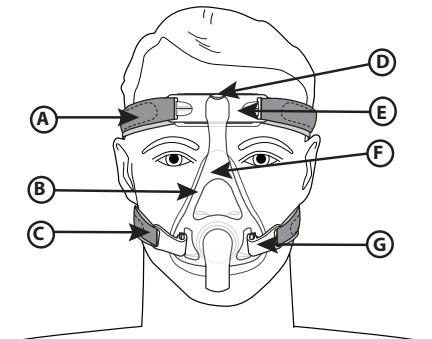
Figure 1

A	Top Headgear Straps
B	Integrated Cushion/Faceplate
C	Bottom Headgear Straps
D	Push Button Adjuster
E	Forehead Pad Support
F	Frame with Elbow
G	Headgear Clips

Contraindications

This mask may not be suitable for use on patients with the following conditions: glaucoma, recent eye surgery or dry eyes, hiatal hernia, excessive reflux, impaired cough reflex, impaired cardiac sphincter function; or on patients unable to remove the mask by themselves.

Before Use Read and Understand the Instructions Completely

- Hand wash the mask.
- Wash your face. Do not use moisturizer/lotion on your hands or face.
- Inspect the mask and replace it if the cushion has hardened or is torn, or if any parts are broken.
- Verify that the therapy device, i.e., ventilator, including the alarms and safety systems, has been validated prior to use.
- Verify therapy device pressure(s).

Cleaning Instructions

Hand wash the mask before first use and daily. The headgear should be hand washed weekly, or as needed. The headgear does not need to be removed for daily cleaning of the mask.

1. Hand wash mask in warm water with liquid dishwashing detergent.

2. Rinse thoroughly. Air dry completely before use. Make sure the mask is dry before use. Lay the headgear flat or line dry. Do not place the headgear into the dryer.

⚠ Caution: Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.

⚠ Caution: Any deviation from these instructions may impact the performance of the product.

⚠ Caution: Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace any components as necessary.

Institutional Disinfection

For **multi-patient use** in the hospital/institutional environment, use the **Disinfection Guide** to reprocess the mask between patients. These instructions can be obtained by visiting us online at www.philips.com/respironics or by contacting Philips Respiration Customer Service at 1-800-345-6443 (USA or Canada) or at 1-724-387-4000.

Dishwashing Instructions

In addition to hand washing, the mask may be cleaned in the dishwasher once a week.

⚠ Caution: Use a mild liquid dish washing detergent only to wash the mask.

1. Remove the headgear. Do not wash the headgear in the dishwasher.
2. Separate the cushion and the faceplate and wash in the top shelf of the dishwasher.
3. Air dry. Make sure the mask is dry before use.
4. Inspect all parts for damage or wear; replace any parts that have visibly deteriorated (cracking, crazing, tears, etc). Replace the mask if the cushion becomes hardened.

Philips Respiration System One Resistance Control

This symbol  may appear on your Philips Respiration therapy device or mask.

The System One setting is not applicable to this mask, or any masks requiring a separate/additional exhalation device.

Achieving the Right Fit

Before putting on the mask:

1. **Headgear clips** - Grip and twist away from the frame to disconnect.

2. **Headgear** - Loosen to a large setting.

Adjusting the mask:

1. Pull the mask over your head. Hold the mask against your face.

2. **Headgear clips** - Press into place to connect (Figure 2).

3. **Headgear** - Pull back the tabs to evenly adjust the top and bottom straps (Figure 3).

4. **Forehead adjuster button** - Press and hold to slide the frame toward or away from your face to adjust pressure (Figure 4). This will decrease or increase the pressure on the bridge of your nose and chin.

The mask should rest comfortably against your face. The forehead pad will touch your forehead lightly. The bottom of the mask cushion should rest just above your chin with your mouth slightly open.

Using the mask:

1. Connect the flexible tubing (included with your CPAP or bi-level device) to the elbow on the mask.

2. Lie down. Turn the therapy device on. Breathe normally. Keep your mouth slightly open.

3. Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments. Some air leaking is normal.

Comfort Tips

- The most common mistake is overtightening the headgear. The headgear should fit loose and comfortable. If your skin bulges around the mask or if you see red marks on your face, loosen the headgear.
- Adjust the bottom headgear straps to reduce leaks at the sides of your mouth.

Removing the Mask

Disconnect the headgear clips and pull over the top of your head (Figure 5).

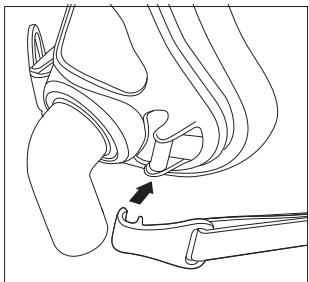
Figure 2

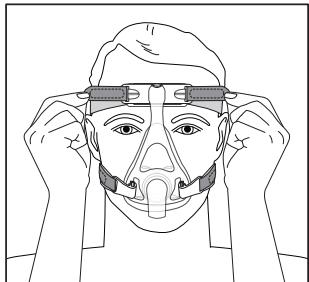
Figure 3

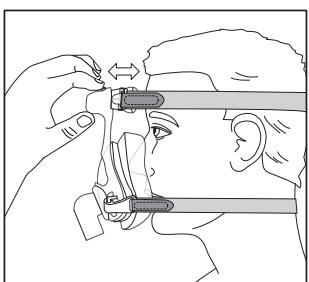
Figure 4

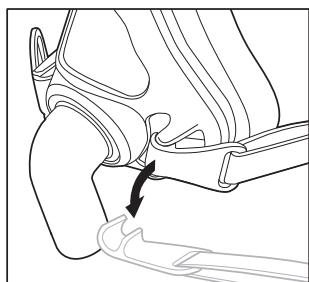
Figure 5

Disassembly of the Mask

1. **Headgear** - Remove from the forehead pad support and headgear clips.
2. **Forehead pad** - Slide off the track of the forehead pad support (Figure 6).
3. **Mask Cushion** - Grasp and pull it off the mask frame (Figure 7).

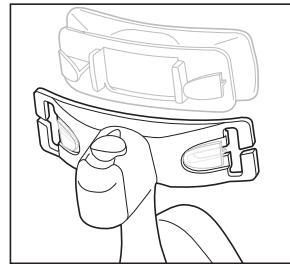
Figure 6

Assembly of the Mask

1. **Forehead pad** - Slide on the track of the forehead pad support.
2. **Mask Cushion** - Use your thumbs to press the cushion past the retaining tabs on the elbow and press into place. (Figure 8).
3. **Headgear** - Thread the top straps through the forehead pad support, and the bottom straps through the headgear clips. Push the headgear clips onto the mask frame.

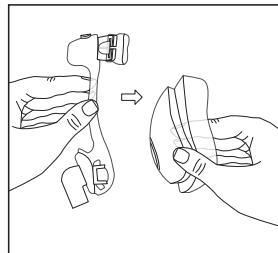
Figure 7


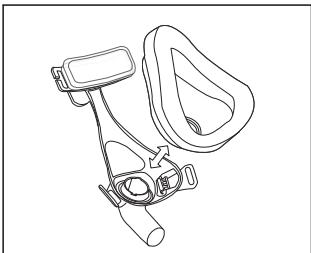
Figure 8

Specifications

⚠ Warning: The technical specifications of the mask are provided for your healthcare professional to determine if it is compatible with your CPAP or bi-level therapy device. If used outside these specifications, or if used with incompatible devices, the mask may be uncomfortable, the seal of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect device function.

Pressure Flow Curve

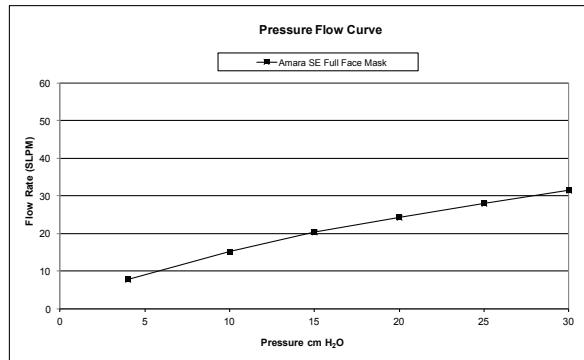Resistance

Drop in Pressure at

	50 SLPM	100 SLPM
All sizes	0 cm H ₂ O	0.2 cm H ₂ O

Deadspace Volume

P	238 ml
S	245 ml
M	256 ml
L	267 ml

Sound Levels

A-weighted Sound Power Level	19 dBA
A-weighted Sound Pressure Level at 1 m	11 dBA

Disposal

Dispose of in accordance with local regulations.

Storage Conditions

Temperature: -4° to 140° F (-20° to +60° C)
Relative Humidity: 15% to 95% non-condensing

Limited Warranty

Respironics, Inc. warrants that its mask systems (including mask frame and cushion) (the "Product") shall be free from defects of workmanship and materials for a period of ninety (90) days from the date of purchase (the "Warranty Period").

If the Product fails under normal conditions of use during the Warranty Period and the Product is returned to Respironics within the Warranty Period, Respironics will replace the Product. This warranty is nontransferable and only applies to the original owner of the Product. The foregoing replacement remedy will be the sole remedy for breach of the foregoing warranty.

This warranty does not cover damage caused by accident, misuse, abuse, negligence, alteration, failure to use or maintain the Product under conditions of normal use and in accordance with the terms of the product literature, and other defects not related to materials or workmanship.

This warranty does not apply to any Product that may have been repaired or altered by anyone other than Respironics. Respironics disclaims all liability for economic loss, loss of profits, overhead, or indirect, consequential, special or incidental damages which may be claimed to arise from any sale or use of the Product. Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

THIS WARRANTY IS GIVEN IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN ADDITION, ANY IMPLIED WARRANTY, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IS SPECIFICALLY DISCLAIMED. SOME JURISDICTIONS DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THE ABOVE LIMITATION MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS UNDER LAWS APPLICABLE IN YOUR SPECIFIC JURISDICTION.

To exercise your rights under this limited warranty, contact your local authorized Respironics, Inc. dealer or Respironics, Inc. at 1001 Murry Ridge Lane, Murrysville, Pennsylvania 15668, USA, or Respironics Deutschland GmbH & Co KG, Gewerbestraße 17, 82211 Herrsching, Germany.

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