**PHILIPS** 

RESPIRONICS

# Trilogy



# Accessory guide



# Table of contents

- 4 Power options
- 6 Data and connectivity
- 8 Carry case and stand
- 10 Circuits and circuit accessories
- 18 Trach adapters
- 19 Exhalation accessories
- 20 Mouthpiece ventilation
- 21 Monitoring and alarms
- 22 Oximetry
- 23 Nurse call cables
- 24 Connectors
- 24 Circuit and device filters
- 25 Port blocks
- 26 Humidification
- 27 Miscellaneous

### **Power options**

# Continuous support, freedom and mobility with an array of Trilogy battery options

As a clinician, you want to make it possible for your patients to go to school and work, attend a ballgame, have lunch with a friend or just visit at grandma's house. Weighing only 11 lbs (5 kgs), and featuring a unique system of internal and easy-to-swap detachable batteries, Trilogy ventilators give patients and their families a new level of freedom and mobility. Choose from a variety of battery options to enhance your Trilogy experience.

#### Respiratory Li-ion battery kit

Our respiratory battery kit provides more than eight hours of battery life<sup>1</sup> and easily attaches to a wheelchair with the carrying case's attachment strap. The compact battery pack, itself, weighs just 2.25 lbs, meets FAA requirements.<sup>2</sup> Plus, the lithuim ion battery is environmentally-friendly.

#### The kit includes

- Li-ion battery pack
- Li-ion battery, Trilogy cable
- Li-ion battery, BiPAP device cable (A30/A40, Canada only)
- Li-ion battery case
- Li-ion battery case attachment strap
- Travel supply, 65W, DreamStation, RoHS

#### Specs

- Recharge time: 3 hours
- Weight: 2.25 lbs
- Dimensions: 1.25" (3.175 cm) x 5.25" (13.335 cm) x 9.0" (22.86 cm)
- Capacity: 90W
- Life cycles: >=70% capacity after 1,000 cycles<sup>3</sup>
- Respiratory battery use run-time test conditions for Trilogy: S/T mode, IPAP 15 cm H20, EPAP 5 cm H20, BPM 12, inspiratory time 0.8 s, passive, compliance 20, resistance 20.
- 2. For airline use. Complies with RTCA/DO-160G Section 21, category M.
- 3. Cycles equal charge/discharge.



#### Universal battery kit and accessories

Part number	Description
1124010	Universal battery kit, North America
1126944	Universal battery kit, International
1126945	Universal battery kit, Japan
1121737	Universal battery cable for Trilogy
1126947	Universal battery case
1126941	Universal battery attachment strap



**Detachable battery pack** A fully-charged, detachable battery will supply at least three hours of usage under normal usage conditions. Hot-swappable on the go.



Detachable battery charger Allows two detachable batteries to be charged at the same time.



External battery cable



Automotive DC power adaptor



Power cord

#### Detachable battery accessories

Part number	Description
1043570	Trilogy detachable battery pack
1072182	Detachable battery charger

#### Trilogy power cords

Part number	Description
1069219	Power cord, North America, 14' (4 m)
1044760	Power cord, North America, 8' (2.4 m)
1134644	Power cord, Argentina, 8' (2.4 m)
1044765	Power cord, Australia, 8' (2.4 m)
1080737	Power cord, Brazil, 8' (2.4 m)
1062054	Power cord, China, 8' (2.4 m)
1044763	Power cord, Europe, 8' (2.4 m)
1044762	Power cord, India, 8' (2.4 m)
1044764	Power cord, Japan, 8' (2.4 m)
1044761	Power cord, UK, 8' (2.4 m)
1047295	External battery cable
1067424	Automotive DC power adaptor

**Data and connectivity** 

Taking patient monitoring to the next level

1111

#### Care Orchestrator remote patient monitoring system

Our monitoring and management system is made up of the Bluetooth-enabled Trilogy ventilator, QCL 2net Hub and Care Orchestrator. This solution delivers actionable patient information directly to care teams, via smartphone, tablet or PC. With Care Orchestrator, everyone can work together to make fast, informed clinical decisions, help identify which patients are at risk and provide timely care to those who need it.

#### **Care Orchestrator offers**

- Customizable daily detail reports, statistics and patterns of usage
- Prescription information
- Health management by exception with tasks/alerts at both the company- and patient-specific levels
- Data uploads via remote connectivity and SD card
- Learn more at www.philips.com/careorchestrator



Qualcomm Life 2net Hub Remote monitoring with Care Orchestrator

#### DirectView patient management software

There are times when patients require changes in therapy. Trilogy DirectView reporting software streamlines reporting to help you identify adjustments necessary to maintain effective treatment. It also allows for prescription management through the use of a SD card.

#### **DirectView offers**

- Breath-by-breath waveforms
- Short- and long-term trends
- Statistics and patterns of usage
- Fully-customizable and easily-exportable reports



SD card and case

Part number	Description	Box quantity
1127528	Qualcomm Life 2net Hub	1/pk
1051801	SD card and case	1/pk
1047300	SD card reader	1/pk
1041151	DirectView SD card mailer	10/pk
	DirectView software package (online download only at my.respironics.com)	

## **Carry case and stand**

# Enhanced portability

With Trilogy's in-use bag and mount stand for use at the bedside, patients have more mobility and freedom.





Travel carrying case

In-use carrying case

Carrying cases			
Part number	Description		
1033709	Travel carrying case		
1040420	In-use carrying case		

#### Home roll stand and accessories

Part number	Description	Box quantity
1047410	Roll stand (includes mounting plate)	1/pk
1094043	Circuit support arm	1/pk
1074004	Roll stand replacement mount plate	1/pk
1074137	Replacement screw for mount plate	4/pk
1009753	Mounting kit, F&P humidifier with water bag holder (For international sales only)	1/pk
1002230	Humidifier mounting dock for roll stand	1/pk
1002229	Bracket for interfacing with mounting dock (1002230)	1/pk
311107	Tube hanger, single limb, 22 mm	1/pk
332497	Support arm, three section	1/pk
1052863	C-clamp bracket for water bag pole mount	1/pk





Hospital roll stand



E-cylinder mount



Cylinder mount



CoughAssist bracket

#### Hospital roll stand and accessories

Part number	Description
1126379	SPRS-2 roll stand, height 111 cm, with base plate
1131462	SPRS-2 roll stand, height 104 cm, with base plate
1131463	SPRS-2 e-cylinder mount kit, 4.25" (108 mm)
1132041	SPRS-2 cylinder mount kit, 5.5" (140 mm)
1132038	SPRS-2 CoughAssist bracket, CA, T70

Not all products are available in all markets.

## **Circuits and circuit accessories**

# For every patient and every need

Our family of circuits is designed to deliver quality respiratory care to pediatric and adult patients with a wide variety of needs.



Active exhalation valve with PAP assembly



Proximal flow sensor



Active exhalation valve with PAP assembly (universal porting block)



Active flow exhalation valve

Part number	Description	Box quantity
1065659	Active exhalation valve with PAP assembly	1/pk
1061398	Trilogy 200, universal active PAP tube adapter	1/pk
1050408	Trilogy 200, proximal flow sensor	1/pk
1050534	Trilogy 200, proximal flow sensor	10/pk
1075459	Active exhalation valve with PAP assembly (universal porting block)	1/pk
1049503	Active flow exhalation valve	1/pk

\*Pediatric patient ≥ 5kg

#### 22 mm passive circuits

Description	Box quantity
22 mm passive circuit with fixed exhalation port, non-heated	10/pk
22 mm passive circuit with fixed exhalation port, non-heated	20/pk
22 mm passive circuit with swivel design exhalation port, non-heated	10/pk
22 mm passive circuit with swivel design exhalation port, non-heated	20/pk
22 mm passive circuit, heated wire	10/pk
22 mm passive circuit, heated wire	20/pk
22 mm passive circuit with water trap	10/pk
22 mm passive circuit with water trap	20/pk
	<ul> <li>22 mm passive circuit with fixed exhalation port, non-heated</li> <li>22 mm passive circuit with fixed exhalation port, non-heated</li> <li>22 mm passive circuit with swivel design exhalation port, non-heated</li> <li>22 mm passive circuit with swivel design exhalation port, non-heated</li> <li>22 mm passive circuit, heated wire</li> <li>22 mm passive circuit, heated wire</li> <li>22 mm passive circuit with water trap</li> </ul>

#### 22 mm active circuits

Part number	Description	Box quantity
1073223	22 mm active circuit, non-heated	10/pk
1073232	22 mm active circuit, non-heated	20/pk
1073222	22 mm active circuit, heated wire	10/pk
1073231	22 mm active circuit, heated wire	20/pk

#### 15 mm passive circuits

Part number	Description	Box quantity
1073225	15 mm passive circuit, heated wire	10/pk
1073234	15 mm passive circuit, heated wire	20/pk
1069445	15 mm passive circuit with fixed exhalation port, with water trap	10/pk
1069446	15 mm passive circuit with fixed exhalation port, with water trap	20/pk
1073224	15 mm passive circuit with swivel design exhalation port, with water trap	10/pk
1073233	15 mm passive circuit with swivel design exhalation port, with water trap	20/pk

#### 15 mm active circuits

Part number	Description	Box quantity
1073221	15 mm active circuit, heated wire	10/pk
1073230	15 mm active circuit, heated wire	20/pk
1073220	15 mm active circuit with water trap	10/pk
1073229	15 mm active circuit with water trap	20/pk

#### **Reusable circuits**

Part number	Description	Box quantity
1052066	22 mm passive reusable circuit, non-heated	1/pk
1052064	22 mm active reusable circuit, non-heated	1/pk
1068837	Trilogy 200 22 mm active w/flow sensor reusable circuit, non-heated	1/pk
1052127	15 mm passive reusable circuit, non-heated	1/pk
1052065	15 mm active reusable circuit, non-heated	1/pk
1068838	Trilogy 200 15 mm active w/flow sensor reusable circuit, non-heated	1/pk

\* Active flow circuits can provide leak estimation and direct measurement of the exhaled gas. They are suitable for advanced flow triggering and are highly tolerant of leaks. Please refer to the Trilogy Series Instructions for Use for more detailed information.

## Circuit details

#### 22 mm passive circuit with fixed exhalation port



#### 22 mm passive circuit with swivel design exhalation port



#### 22 mm passive circuit, heated wire



#### 22 mm passive circuit with water trap



#### 22 mm active circuit



#### 22 mm active circuit, heated wire



#### 15 mm passive circuit, heated wire



#### 15 mm passive circuit with fixed exhalation port, with water trap



## 15 mm passive circuit with swivel design exhalation port, with water trap



#### 15 mm active circuit with water trap



#### 15 mm active circuit, heated wire



## **Trach adapters**

# Customized care



**Flexible trach adapter with 22 mm connection** Connects to exhalation ports used with the passive circuit.



#### Flexible trach adapter with 15 mm connection

Connects to the flow sensor of the active exhalation valve.

Part number	Description	Box quantity
1073902	Flexible trach adapter with 22 mm connection	10/pk
1073904	Flexible trach adapter with 22 mm connection	20/pk
1073863	Flexible trach adapter with 15 mm connection	10/pk
1073903	Flexible trach adapter with 15 mm connection	20/pk

## **Exhalation accessories**

# Greater comfort, ease and accessibility



Whisper swivel II



Disposable fixed exhalation port



Disposable filtered exhalation port with CAP



Disposable swivel passive exhalation port

Part number	Description	Box quantity
332113	Whisper swivel II	1/pk
312149	Disposable fixed exhalation port	10/pk
1065775	Disposable filtered exhalation port with CAP	10/pk
1084428	Disposable swivel passive exhalation port	10/pk
1084429	Disposable swivel passive exhalation port	20/pk

## **Mouthpiece ventilation**

# Giving patients freedom and flexibility

Our Mouthpiece ventilation and circuit support system help ventilation patients breathe on demand.

#### Mouthpiece ventilation (MPV)

A 'kiss' trigger with signal flow technology detects when a patient engages and disengages from the mouthpiece to deliver on-demand ventilation. Combine mouthpiece ventilation with an optional MPV support system to enhance patients' ease of use.



#### MPV disposable circuit (1102866)

Each system contains a bacteria filter, 6' (15 mm) tube, miniature flex tube, angled mouthpiece, straw mouthpiece and all necessary adaptors.



#### MPV circuit support system

This interlocking support system combines with Trilogy's breathing circuit to provide easy usage for patients. The support system can be attached to various surfaces in many configurations with the use of a clamp designed for use on flat or rounded surfaces. The interlocking support circuit can be adjusted to enhance comfort and accessibility.

Each system includes a clamp, 2' bendable tube holder and one MPV disposable circuit.

MPV circuit support system (1102862)

Part number	Description	Box quantity
1102866	MPV disposable circuit	10/pk
1118840	MPV replacement straw kit	10/pk
FC06566	Angled replacement mouthpiece, 22 mm	1/pk
1102862	MPV circuit support system	1/pk

## **Monitoring and alarms**

# Alert to changing conditions and events

Our remote alarm provides an audible tone and visible flashing red LED to alert caregivers of an alarm condition on Trilogy from up to 61 m (200').

#### The key features of the remote alarm include

- A warning when Trilogy alarms alert or its 9-v battery backup is low
- Visual indicators for alarms, low battery and AC power
- Simple interface (on/off and self-test buttons)
- Automatic self-test at startup and self-test any time
- Standard 9-v battery backup with a low-battery alarm



Remote alarm Includes 50' (15 m) RJ9 cable, power supply and 9-v battery

#### **Monitoring and alarms**

Part number	Description
1118941	Remote alarm, North America
1120632	Remote alarm, International

#### **Remote alarm accessories**

Part number	Description
1120633	Power supply, North America
1120634	Power supply, International
1127161	Power supply, UK
1114933	Remote alarm cable, RJ9, 50' (15 m)
1114934	Remote alarm cable, RJ9, 200' (61 m)
34046	Remote alarm stand

Not all products are available in all markets.

## Oximetry

# Convenient way to measure



Nonin SpO<sub>2</sub>

Oximetry interface kit		
Kit includes interface adapter,		
SD card and mailer, carry case		

Part number	Description
1080964	Oximetry interface kit
1122672	Nonin SpO <sub>2</sub>

### **Nurse call cables**

# Connect and communicate



Nurse call adaptor cable (1045290)



NC nurse call cable (1069979 / 1069980)



RJ9 nurse call cable assembly adapter (1080249)



Isolated RS232 serial cable (1046972)



RJ9 to 2.5 mm 3" female adapter cable (1080250)



RJ9 to BNC cable (1079886)



\_\_\_\_\_

Stereo jack to BNC cable (1045291)



RJ9 nurse call cable alarm, closed (1080247 / 1080248)

Part number	Description
1045290	Nurse call adaptor cable
1069979	NC nurse call cable, 0.25" to 2.5 mm
1069980	NO nurse call cable, 0.25" to 2.5 mm
1046972	Isolated RS232 serial cable
1080247	RJ9 to 0.25" nurse call cable alarm, closed
1080248	RJ9 to 0.25" nurse call cable alarm, open
1080249	RJ9 nurse call cable assembly adapter
1080250	RJ9 to 2.5 mm 3" female adapter cable
1079886	RJ9 to BNC cable
1045291	Stereo jack to BNC cable

## Connectors

# Choose soft- or hard-flex tubes and accessory adapters

Part number	Description
C06492	Flex tube, 12", 22 mm x 22 mm
C06495	Flex tube, 8", 22 mm x 22 mm
1006042	Flex tube, 5.5", 22 mm x 22 mm
1006043	Flex tube, 2", 22 mm x 15 mm, tapered
1004968	Flex tube, 6", 22 mm x 15 mm, tapered
C06483	Flex tube, 3.5", 22 mm x 15 mm, tapered
C06334	Connector 22 mm x 22 mm ID, female
1044191	Adapter 22 mm x 15 mm OD
1044186	Adapter 22 mm x 22 mm OD



Adapter 22 mm x 15 mm OD (1044191)



Connector 22 mm x 22 mm ID, female (C06334)

### **Circuit and device filters**

# Cleaner air with each breath





Bacteria filter

Part number	Description	Box quantity
1029330	Foam filter	2/pk
C06418	Bacteria filter, 22 mm, single-use	1/pk
C06417	Bacteria filter, 22 mm, single-use	50/pk

Foam filter

## **Port blocks**

# Can easily be swapped to meet your circuit preference

Choose the best available circuit and interface for your patients with the ability to use either an active or passive exhalation port. Trilogy's unique passive exhalation port eliminates extra tubing connections, resulting in fewer disconnections.



Passive porting block



Active porting block



Universal porting block

Part number	Description
1040372	Exhalation port block, passive
1054670	Exhalation port block, active
1040370	Universal porting block,Trilogy 200/202

## Humidification

# Have more control over the humidity level



Airlife, pediatric

Airlife, adult

#### HMEs

Part number	Description
C06274	Airlife, adult
C06273	Airlife, pediatric

#### **Replacement humidifier chambers**

Part number	Description	Box quantity
22105	Single-use auto-feed humidification chamber	1/pk
22104	Single-use auto-feed humidification chamber	10/pk
22182	Replacement humidifier chambers for MR410	1/pk
22183	MR250 single-use humidification chamber, adult	1/pk

## Miscellaneous





Oxygen inlet adapter

Test lung

Part number	Description	Box quantity
1040390	Oxygen inlet adapter	2/pk
1021671	Test lung	1/pk
1045587	Inlet airpath assembly	1/pk



© 2017 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice.

www.philips.com/respironics

Caution: US federal law restricts these devices to sale by or on the order of a physician.

RRDPGH RB 10/13/17 MCI 4107938 PN 1134800 1010 Murry Ridge Lane, Murrysville, PA 15668 800-345-6443 • 724-387-4000