

It's as easy as 1, 2, 3...

1. Fill

Simply lift the green lid and pour in the medication. Close the lid and make sure you're sitting up straight, ready for therapy.



2. Treat

Press the on button and breathe in and out through the mouthpiece.*** InnoSpire Go will indicate therapy completion with audible and visible signals - before turning itself off.




3. Clean

After each treatment remove the mouthpiece and rinse it in warm water. At the end of each day wash it in warm soapy water and once a week disinfect it in boiling water.



Find out more

www.philips.com/innospirego

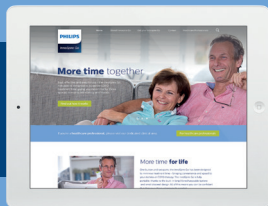

** Using 2.5ml salbutamol
*** An adult mask is available if you find it difficult to use the mouthpiece.



PHILIPS

InnoSpire Go

Portable
mesh nebulizer



More
time
for life


© 2016 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice. Respironics Inc. and Respironics Respiratory Drug Delivery (UK) Ltd are Philips companies.

Respironics Respiratory Drug Delivery (UK) Ltd
Chichester Business Park
City Fields Way
Tangmere
Chichester
PO20 2FT UK

KW 10/20/2016 PN452300
PN 1132090 MCI 4107772
www.philips.com/respironics

More time together

Fast, effective and easy to use, the InnoSpire Go nebulizer is designed to shorten COPD treatment time, giving you more time for those special moments with family and friends.

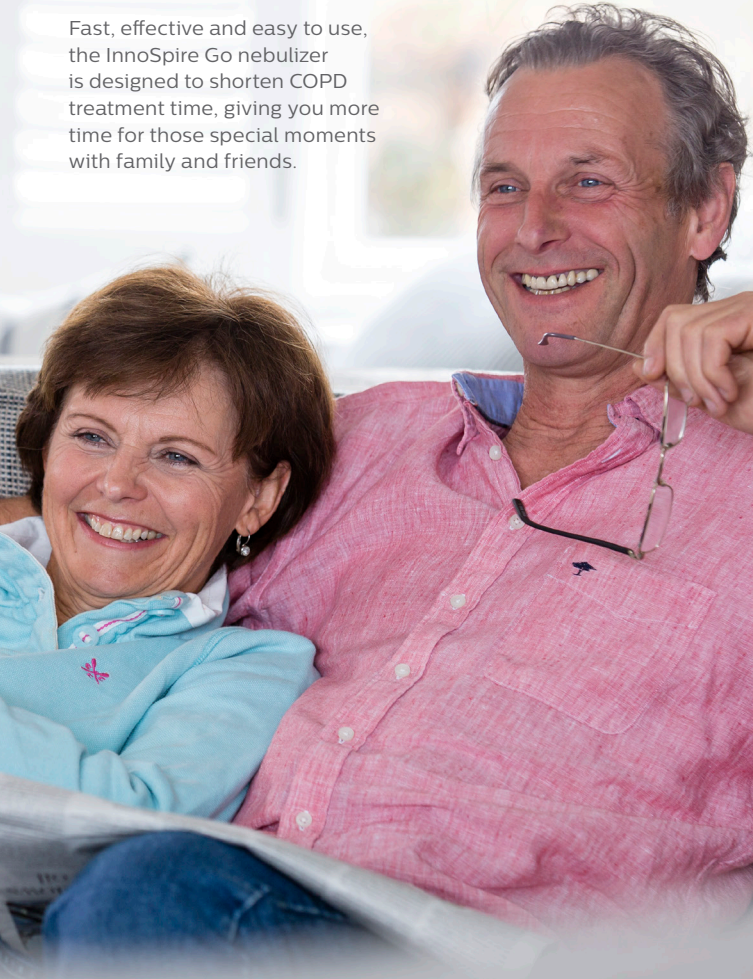

Reduces treatment time by 25%*



Treatment in 4 minutes**

An efficient COPD therapy device that indicates therapy completion using audible and visible signals - before automatically turning off.



Virtually silent

A small, portable and discreet nebulizer that can be used anywhere.



Built-in long life battery


The rechargeable battery provides up to 30 treatments between each charge.



Simple - only two parts

The easily detachable mouthpiece makes the InnoSpire Go simple to use, clean and maintain.***

Find out more
www.philips.com/innospirego

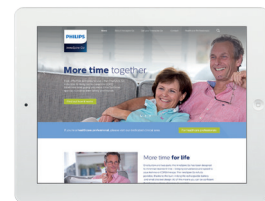

← Actual size →

* Versus predecessor, Aeroneb Go.

** Using 2.5ml salbutamol.

***An adult mask is available if you find it difficult to use the mouthpiece.