


## Launching Stress Echo


1 Press **Setup** or **Protocol** on the control panel keyboard.




2 Select the **Stress** tab at the top of the setup menu.

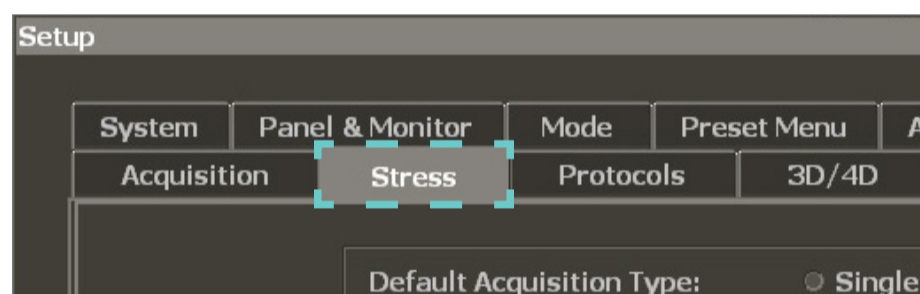

3 Press **Load Defaults**.

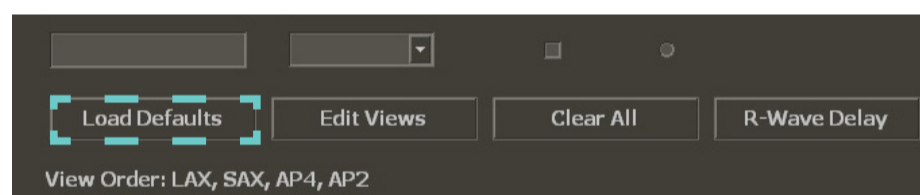

4 Select one of the three default protocols listed in the popup window. Click **OK**.

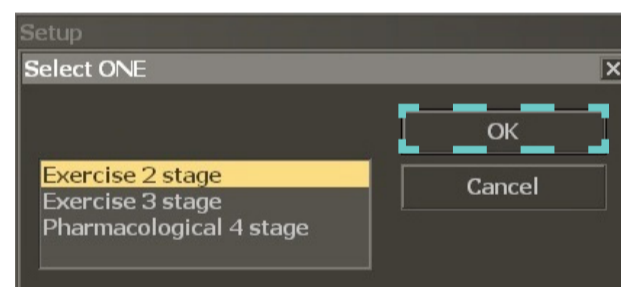

5 Press **Apply** to begin exam with default  
or  
**Save** to create a custom preset

### Custom Presets

Custom presets are only available from the Presets list by pressing the **Preset** button on the console.

Custom preset features...

- 8 stages and views
- Up to 4 multicycles
- Ability to set Start view
- 1 timer



# Performing a Study


## Acquisition

- 1 Acquire images by pressing **Acquire** button on control panel.
- 2 Accept image by pressing **Acquire** again *or* **Enter**

## Multicycle Acquisition

- 1 Press **Enter** to change the view name
  - 2 Apply the **Gain Save** feature
- Note: Multicycle stage can be defined when creating a custom preset.


## Image Indicators

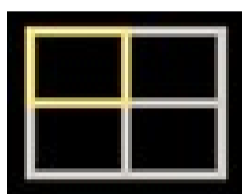


Active image.



At least one image has been selected.

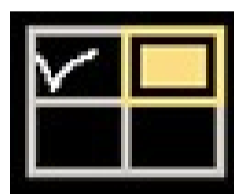

Images were acquired, but not selected.

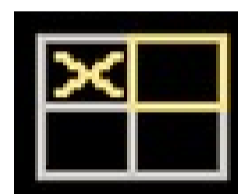

View or stage was skipped.

## Pause Exam

- 1 Press soft key 4 to **Pause** protocol

## Resume Exam

- 1 Press **2D** *or* **Erase**
- 2 Go to **Review** and select patient exam


## Accept Stage

- 1 Press **Enter**
- 2 Using soft key 2 select...  
**Skip View**  
*or*  
**End Stage**

## Selecting Multicycle Images

Multicycle images are stacked behind each view.

- 1 Enable the **Select Multiple** checkbox.
- 2 Navigate between images with the forward and back buttons.
- 3 Select desired images by checking the yellow box in the left corner of each image.

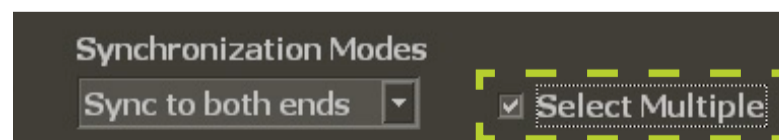

- 4 Click the grid button to return to other views.

**Note:** Only the images that have been selected are exported. All images are saved on the hard drive until the study is deleted.




## Reject an Image

- 1 Press **2D** or **Erase**

## Tips


### Relabel View

- 1 In **Review**, right-click on an image
- 2 Select **Relabel View**
- 3 Select desired label

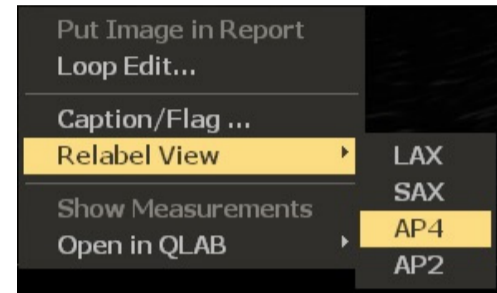

### Compare Pre and Post Images

- 1 In the **Stress Display** menu, select **Display Views**

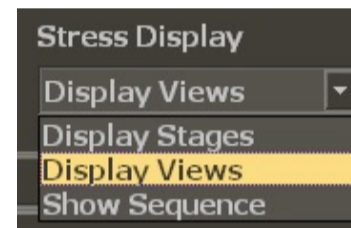

### Compare Mode

- Once the first stage is acquired, soft key 1 will display **Compare**.
- The rest stage will display on the left and current view will display on the right of screen.

### Physio

- ECG beats revert to time if ECG signal is lost
- Must turn **Physio** knob to **External** if connecting to the stress station

### Export

- Only the images that have been selected are exported. All images are saved on the hard drive until the study is deleted.

