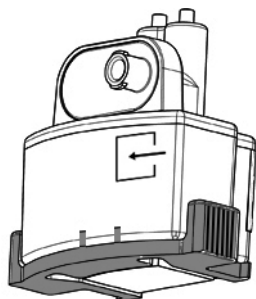
Water Traps

Instructions for Use

REF M1657B

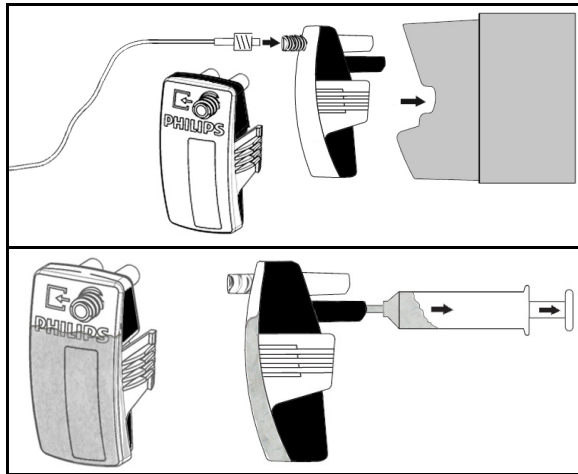
REF 989803191081

PHILIPS

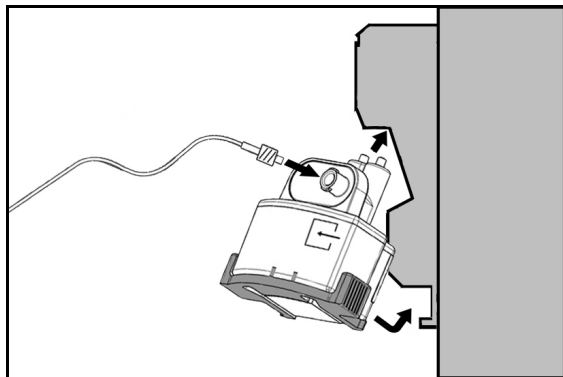
Contents

(BG) Указания за употреба	1
(CS) Návod k použití.....	2
(DA) Brugerhåndbog.....	3
(DE) Gebrauchsanweisung.....	4
(EL) Οδηγίες χρήσης.....	5
(EN) Instructions for Use	6
(ES) Instrucciones de uso	7
(ET) Kasutusjuhend.....	8
(FI) Käyttöohjeet	9
(FR) Manuel d'utilisation.....	10
(HR) Upute za uporabu	11
(HU) Használati útmutató	12
(IT) Istruzioni d'uso	13
(JA) ユーザーズガイド	14
(KK) Пайдалану нұсқаулығы.....	15
(KO) 사용 설명서	16
(LT) Naudojimo instrukcija.....	17
(LV) Lietošanas instrukcija	18
(MK) Инструкции за користење.....	19
(NL) Gebruiksaanwijzing.....	20
(NO) Bruksanvisning	21
(PL) Instrukcja obsługi	22
(PT-BR) Instruções de Uso.....	23
(RO) Instrucțiuni de utilizare	24
(RU) Инструкция по эксплуатации	25
(SK) Návod na použitie	26
(SL) Navodila za uporabo	27
(SR) Uputstvo za upotrebu	28
(SV) Bruksanvisning	29
(TR) Kullanım Talimatları.....	30
(UK) Інструкції з використання	31
(ZH-CN) 使用说明 (书)	32
(ZH-TW) 使用說明	33

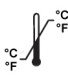
989803191081

M1657B

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Product Description

The Water Trap is a reusable receptacle intended to separate and collect water from a patient's breathing circuit during gas measurement. It prevents water from entering the Gas Analyzer and helps protect the patient and breathing circuit from bacteria and viruses. The Water Trap is restricted to use only by trained medical professionals. *Note: All products described may not be available in all countries.*

Water Trap Models and Compatibility

All information in this Instructions for Use is applicable to two different model Water Traps, except as noted otherwise:

- **M1657B** Water Trap: Compatible for use only with Philips M1013A (G1), M1019A (G5), M1026B, and M1026A #A02/A05 Gas Analyzer instruments.
- **989803191081** Water Trap: Compatible for use only with the Philips 866173 IntelliVue G7m Gas Analyzer instrument.

The Water Trap connects to a patient breathing circuit. Refer to the Philips *Water Trap Accessories Instructions for Use* for information on setting up Water Trap accessories that are available for gas sampling from the patient breathing circuit.

Warnings

Observe all Warnings to prevent damage to the Water Trap and gas measurement components, and to minimize inaccurate gas measurements and possible patient injury:

- Reference the appropriate *Gas Analyzer Instructions for Use* and the *Water Trap Accessories Instructions for Use* for additional product information and Warnings.
- Do not use nebulizers or aerosol in the patient breathing circuit when the breathing circuit is connected to the Water Trap. *Note: Nebulizers may be used only when the Water Trap is disconnected from the patient breathing circuit.*
- Never rinse, clean, or disinfect the *inside* of the Water Trap, sample tubing, or airway adapters.

Installing and Removing the Water Trap

Reference the appropriate Water Trap graphic on page i at the front of this document.

1. Position the Gas Analyzer at or above patient level to avoid collecting condensed water in the breathing circuit.
2. If installing a 989803191081 Water Trap push it straight into the holder at the front of the Gas Analyzer; if installing a M1657B Water Trap push it up and into the holder at the front of the Gas Analyzer. An audible click provides indication that the Water Trap is installed and seated correctly.
3. Connect the Gas Sample Tube from the patient's breathing circuit to the connector on the Water Trap, assuring that the tubing is not kinked. A kinked sample tube could cause an occlusion or leakage.
4. Check that all patient breathing circuit connections are tight to prevent ambient air from mixing with patient gas and causing erroneous readings.
5. To remove either Water Trap, squeeze its fluted side-tabs and pull it away from the Gas Analyzer. Be careful to hold it upright as it disengages from the Gas Analyzer holder to prevent spilling contaminated liquid.

Emptying or Disposing the Water Trap

Warning: Liquid in the Water Trap may be contaminated and must be handled and disposed of with care. Dispose in compliance with local regulations for hospital waste.

- **M1657B Water Trap Disposal:** The M1657B Water Trap cannot be emptied. The M1657B must be disposed of as infectious waste when full, or after 2 weeks maximum use, whichever occurs first.
- **989803191081 Water Trap Emptying or Disposal:** The 989803191081 Water Trap can be emptied and reused as described below for up to 4 weeks maximum use. After 4 weeks of use dispose of the 989803191081 as infectious waste.
 1. Disconnect the sample line from the 989803191081 before emptying to prevent spilling contaminated liquid.
 2. Squeeze the fluted sides of the 989803191081 and pull it away from the Gas Analyzer, being careful to hold it upright as it disengages from its holder.
 3. Insert an empty syringe with a filling volume of at least 20ml without cannula into the lower connector on the back of the 989803191081, as shown on page i at the front of this document.
 4. Extract the water into the syringe, then remove and dispose of the entire syringe as infectious medical waste.
 5. Reinstall the 989803191081 as described in the installation procedure above.

Care and Cleaning

Warnings: Never clean the inside of the Water Trap. Do not sterilize or autoclave the Water Trap.

The outside of either Water Trap may be *wiped* clean and disinfected with Ethanol 70%, Isopropanol 70%, Cidex, Kohrsolin, or Sporidicid. *Note: Do not wipe disinfectant over the opening in the Water Trap sample tube connector.*

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Copyright © 2015
All rights reserved
Böblingen GmbH
Philips Electronics Singapore Private Limited