EKG - sync - Verteiler Box
ECG - sync - Distribution Box

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Art.-Nr.: 2500-00-0-000-00

EKG-sync-Verteiler Box

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
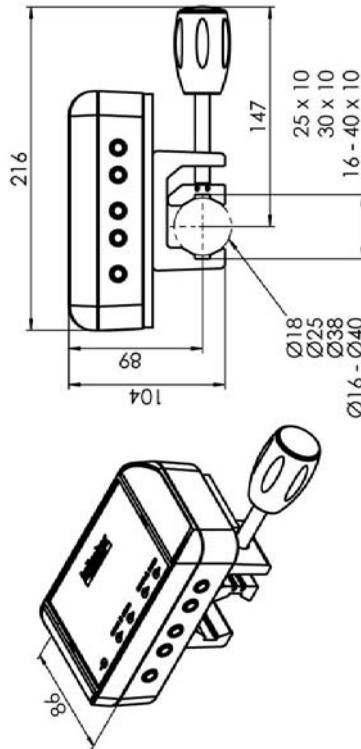
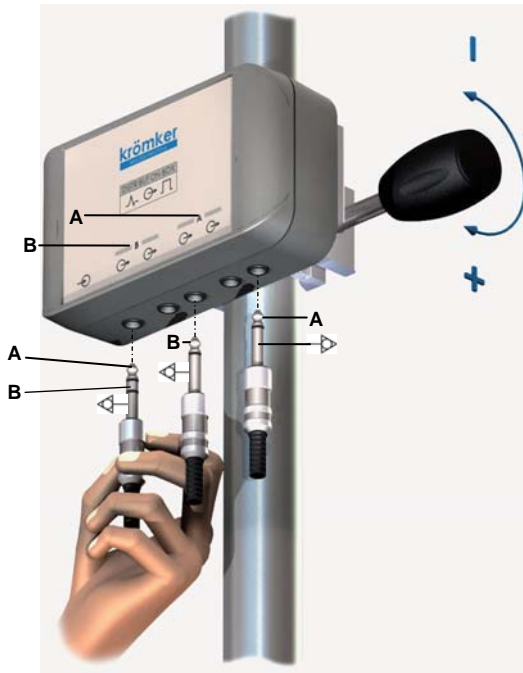
Frontansicht | Front View

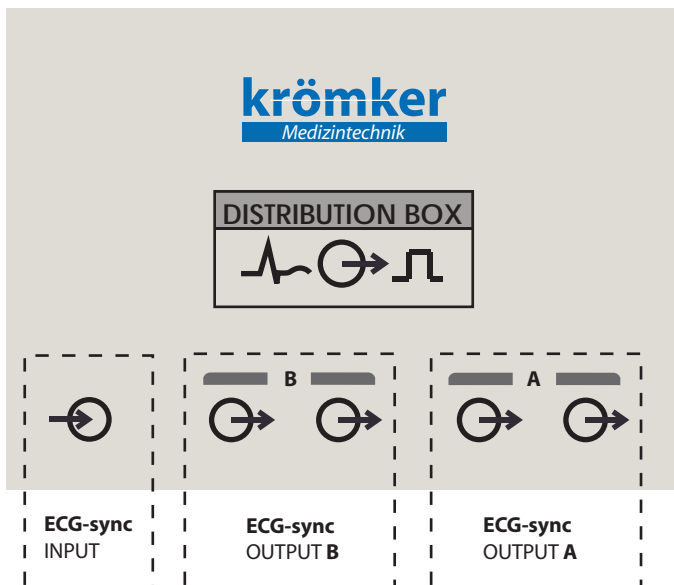ECG-sync Distribution Box

Technical data of the jack chassis socket acc. to JIS 6560, 6,35mm, 3 poles/stree

1. Materials

Body	PPO
Contact	CuZn/CuSn

2. Mechanical data

Insertion force	5-40N
Withdraw force	5-40N
Mating cycles	>5000
Mating with jack plugs KLS 3, KLS 33	

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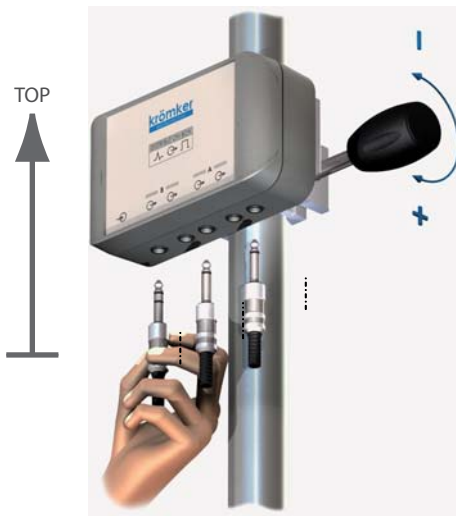
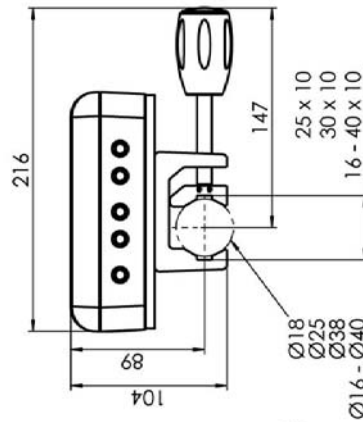
Usage

The input socket of the EKG-Splitter-Box is to be connected with the output socket of a Philips monitor, in particular the ECG-sync-output whereby the signal is then split into 4 identical signals which can be taken off through the four output sockets of the Box.

The output sockets of the box can then be connected to the input sockets of for example:

- Defibrillator
 - CT/MR scanner
 - trans esophageal echocardiography TEE
 - Intra-aortic balloon pump IABP
- thus allowing for ECG-synchronisation or triggering of the external systems.

Desinfektion

Please read and understand the relevant documentation for use before using disinfectants. Wipe and spray disinfection with for example Antifect Liquid or Mikrozyd AF Liquid (Schülke & Mayr) ready to use solutions. Take care to moisten the entire surface (40ml/m² approx). Remove surplus solution by using a colour fast one way cloth. Reaction time for surfaces within hospitals and practices = 5 Minutes.

Hints

1. Mounting

- Note the preferred mounting position
- Ballengriff hand-fest anziehen
- Fasten the handle by use of hand force
- Fasten the handle tight enough to insure that the appliance does not slide on the pole

2. Predictable misuse

- avoid knocking or collisions with the box
- do not overload
- do not use the box as a tray

3. Safety

- Check all sockets for signs of corrosion before every use
- Do not use the system if corrosion is apparent
- Do not use the system if you have a suspicion it may be damaged

4. Starting up

Connect all relevant/required external devices and check the functionality within the network. If all functions of external devices are available then the box is ready for use.