



Trilogy202




Quick start guide

- 1 Overview
- 2 Circuit setup
- 3 Setup screen
- 4 Prescription setup




1 Overview

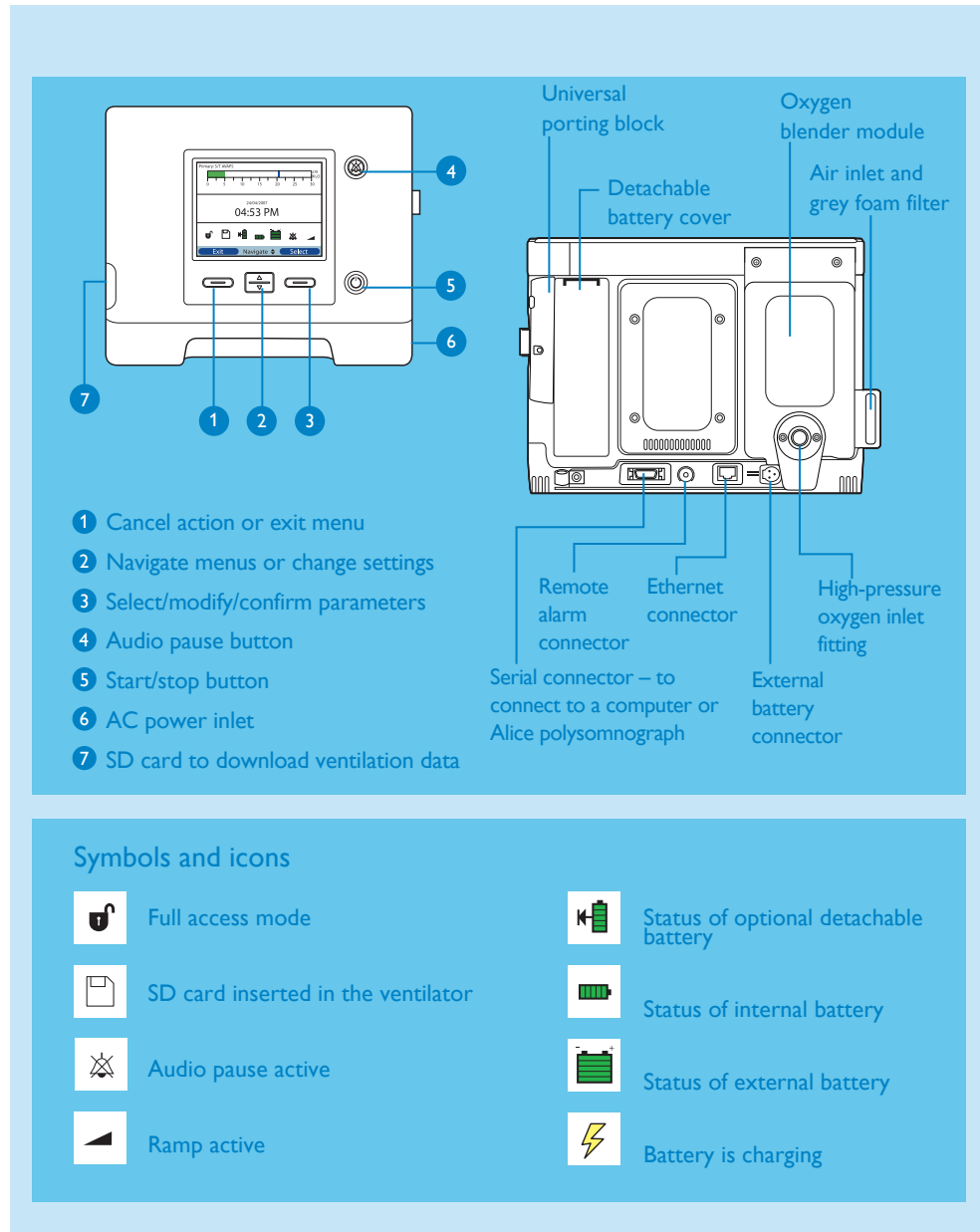

2 Circuit setup


Before operating the Trilogy202, you need to configure the circuit type.

Passive exhalation port circuit

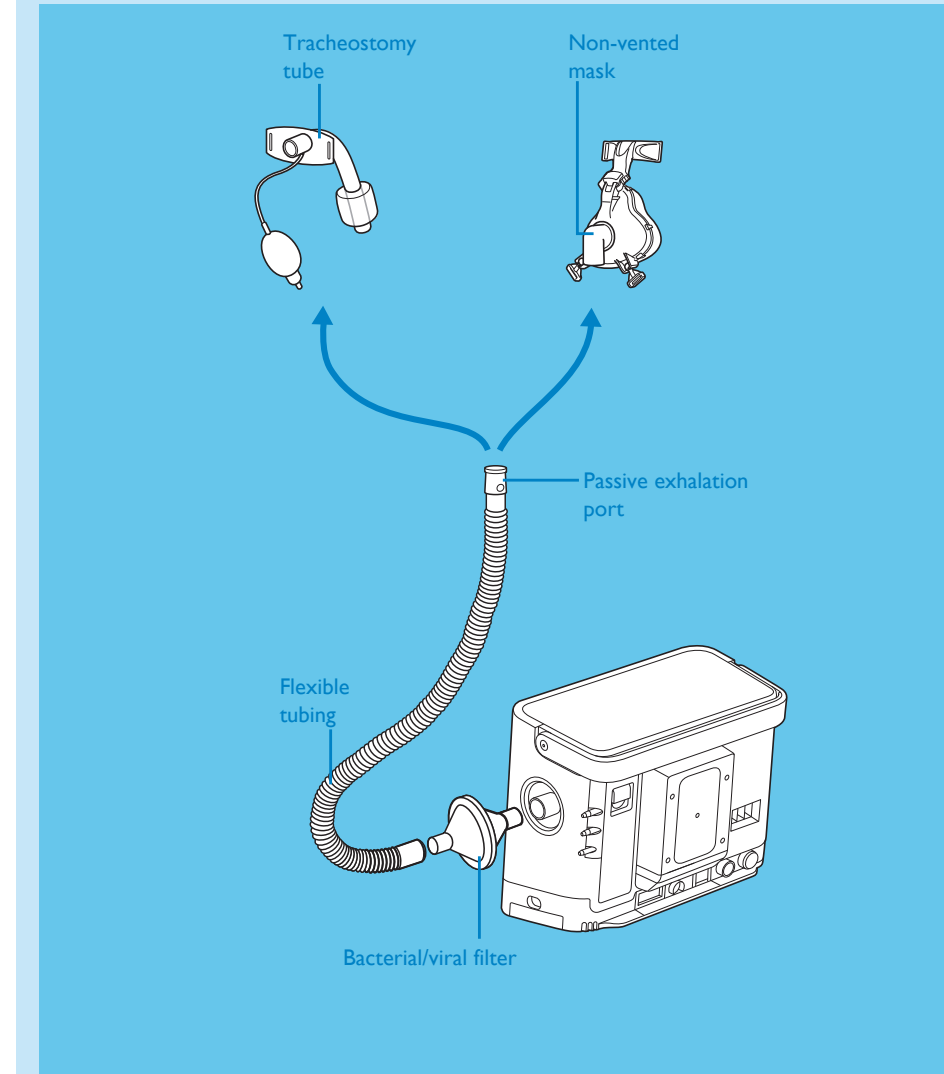


From the Setup screen select "Passive" for circuit type.

Active exhalation valve with proximal airway pressure circuit

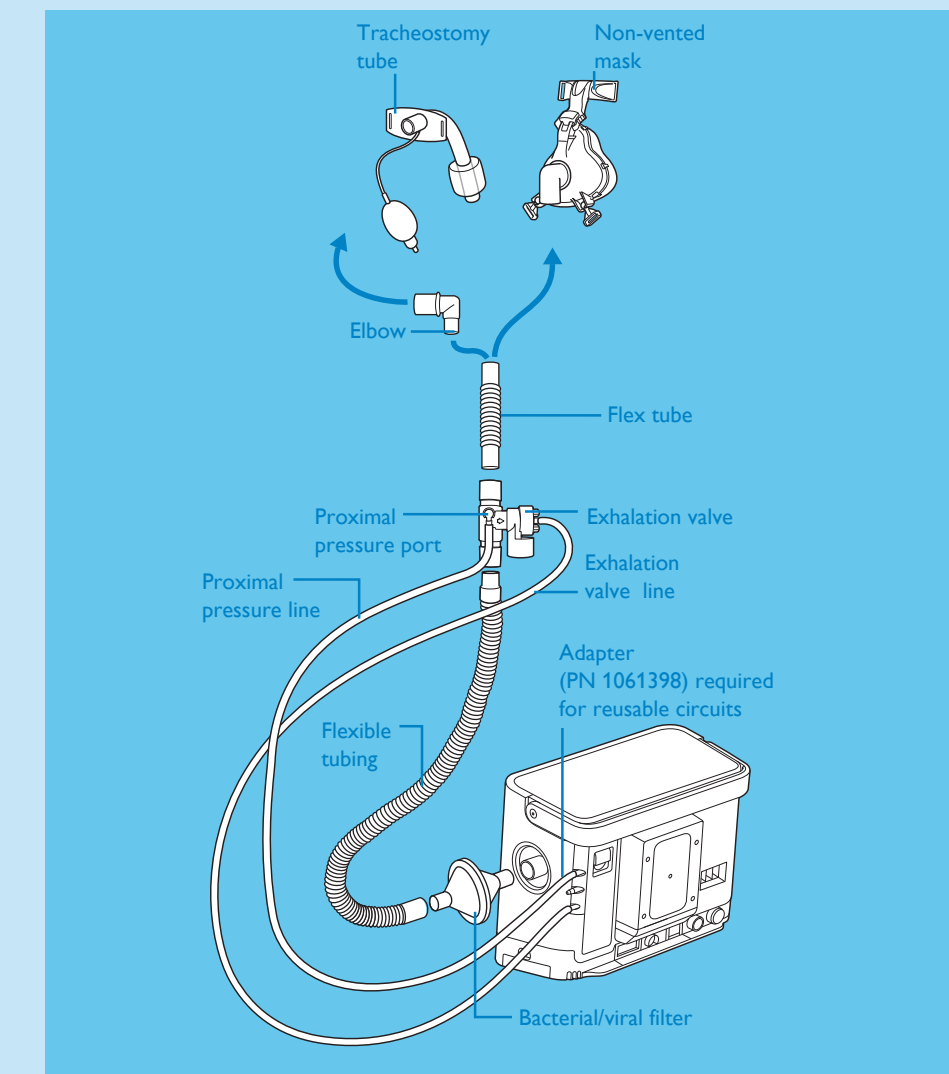

From the Setup screen select "Active PAP" for circuit type.

Active exhalation valve flow sensor circuit

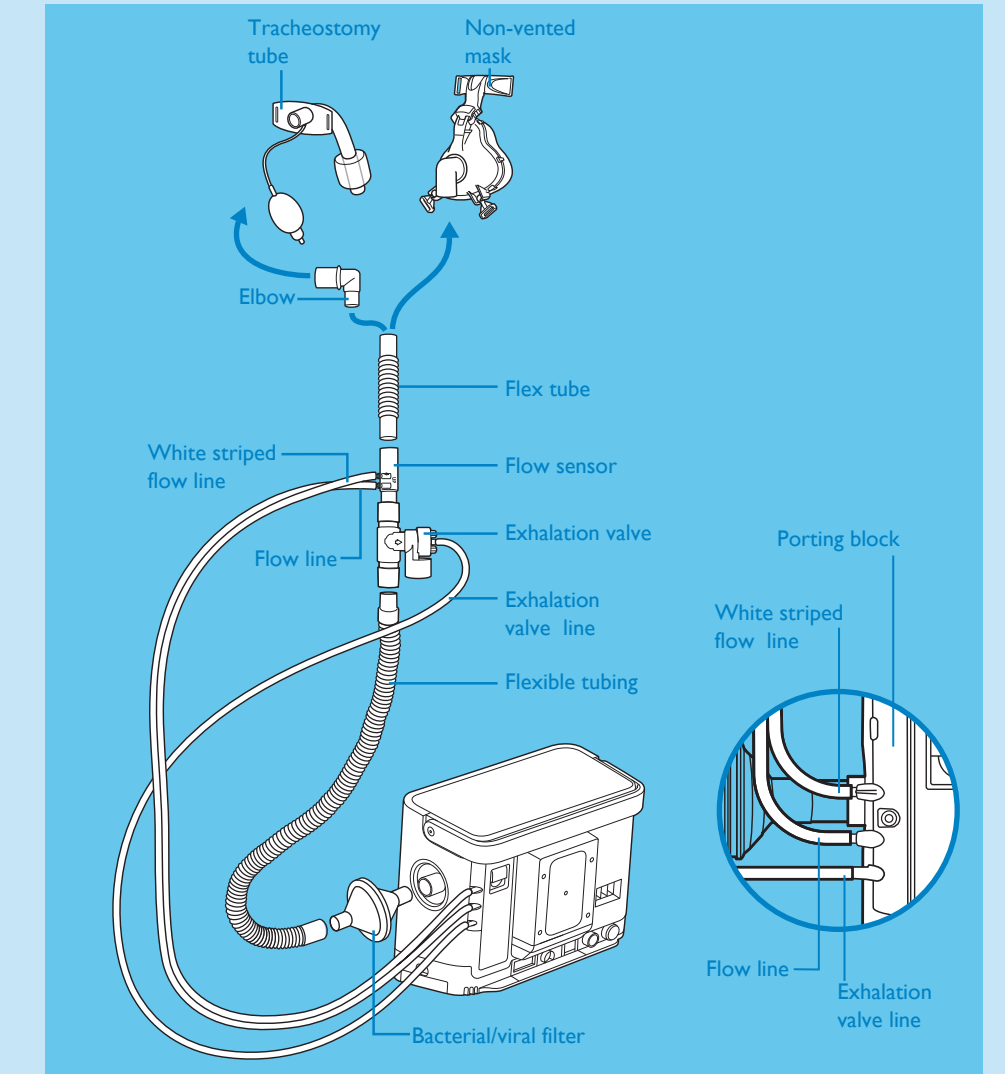


From the Setup screen select "Active flow" for circuit type.


3 Setup screen

Configuring the circuit type (active or passive)


Make sure the Trilogy 202 is turned off before taking the following steps

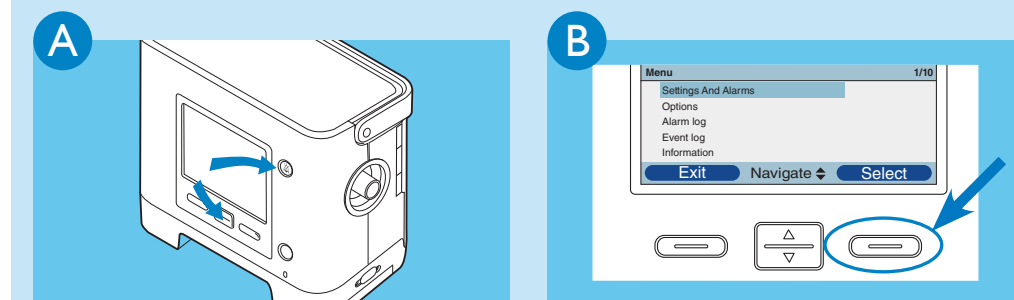


Press and hold and for a few seconds to access the **Setup** screen

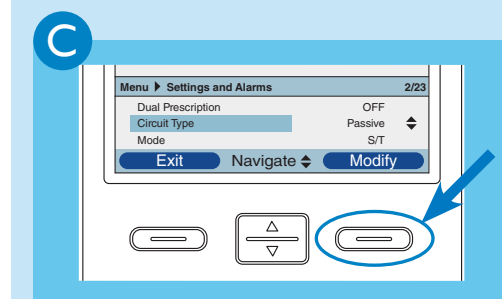

Navigate to **Circuit Type** and press **Modify**

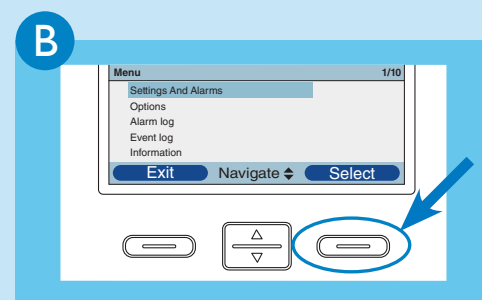


Select **Settings And Alarms**

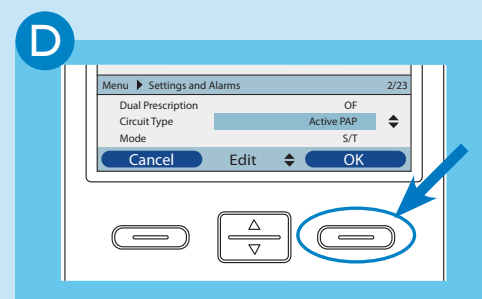


Use to choose **Active PAP**, **Active flow**, or **Passive** and then press **OK** twice to confirm

Setting menu access (May be performed while on or off the patient)


To enable full access to the settings

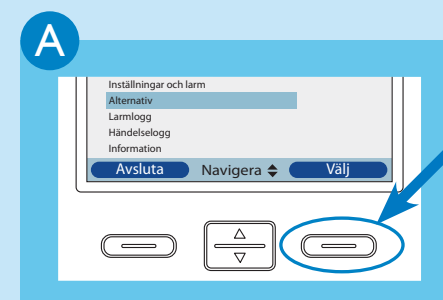


Navigate to **Options** and press **Select**

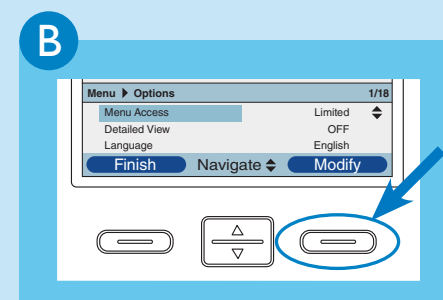


Press **Modify** to enter **Menu Access**

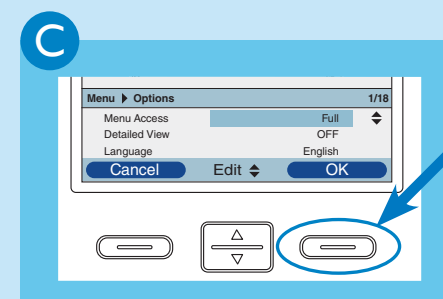

Use to choose **Full** or **Limited Access** and then press **OK** to confirm

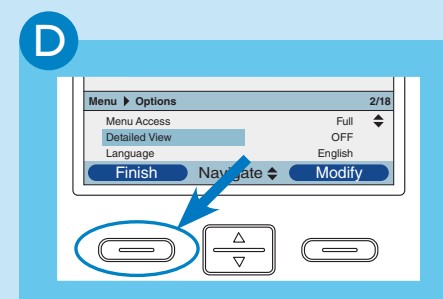

Press **Finish** and **Exit** to return to **Initial Screen**

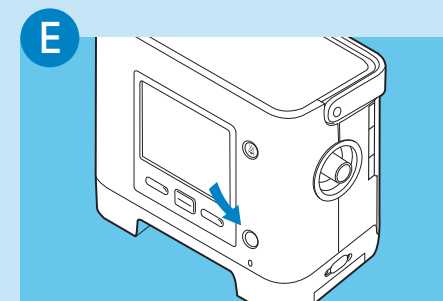

Press to begin therapy or continue navigating through the menu to set up the ventilation parameters

4 Prescription setup

A

From main screen, press **Menu**

B

Select **Settings And Alarms**

C

Scroll up and down using to navigate through the settings and alarms

D

Press **Modify** to change the parameter highlighted in blue

E

Use to change the value or status of setting

F

Press **OK** to validate new setting

G

Press **Finish** and **Exit** to return to the main screen

Dual Prescription

If **Dual Prescription** is enabled, both prescriptions need to be adjusted as previously explained

I

Set both primary and secondary prescriptions

Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

Respironics Deutschland
Gewerbstrasse 17
82211 Herrsching, Germany



1068642
JH 7/20/10