

Motion Biosensors for Dermatology Studies

Measuring Motion, Delivering Insights

Objective nocturnal sleep & scratching event measurement to evaluate the impact of a particular therapy or medication pertaining to Atopic Dermatitis.

Philips New Standard in Sleep & Scratching Measurement

- Reduce time associated with Video scoring and Patient Reported Outcomes (PRO) subjectivity with automated high speed 3D motion data capture.
- Leverage data collection, analysis and insights via a medical device analytical tool - to monitor device utilization and therapy impact on motion and sleep -for direction regarding next phase research decisions.
- Discover more effective therapies for more people, through objective measures of scratching and sleep events.


High resolution 3D motion capture discerns basic movement from scratching.





Atopic Dermatitis Study: 98% Correlation between Data captured through Wearable Motion Biosensor Technology and Video Analysis

Scratching measurement across 24 subjects: Motion BioSensor and Video scored

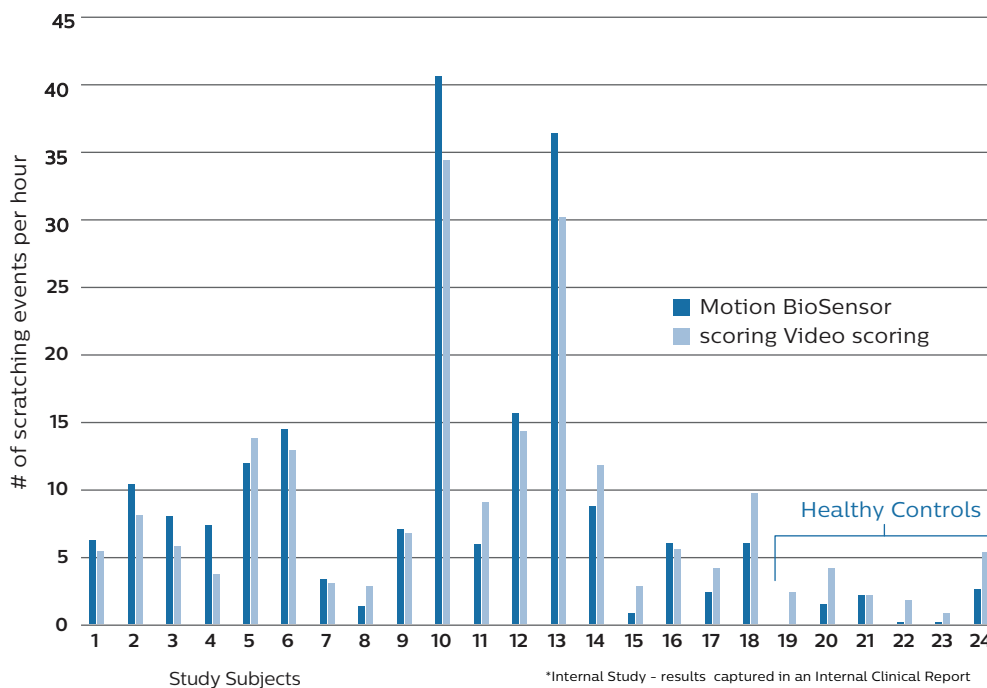

More than a Device

- Advanced proprietary algorithms enable rapid data processing across sleep and wake periods – for multiple clinically relevant data endpoints. (i.e. # of scratching events, duration)
- Statistical data analysis and consultation services provide direction on protocol development and guidance regarding research/trial advancement.
- Actionable intelligence enhances overall research/trial preparation and execution efficiency.

- Philips advanced algorithm enhances efficiency with automated capture of customized data endpoints.
- Philips Motion Biosensor technology may be applicable to other areas of Dermatology, where motion provides relevant clinical insights.

Correlation rate indicates that Motion BioSensor technology offers a minimally invasive and economic alternative to Visual Data Scoring.

Novel Scratching Endpoints:

- # of scratching event(s) per subject
- # of scratching event(s) per hour
- Duration of each scratching event
- Time of each event

Conventional Sleep Endpoints:

- Sleep efficiency
- Wake after sleep Onset (WASO)
- Total Sleep Time

© 2016 Koninklijke Philips N.V. All rights reserved. Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.



Philips Respironics
920 SW Emkay Dr. STE., 100 Bldg. C
Bend, OR 97702 USA

www.philips.com/motionbiosensors

+1 541 598 3800

Toll free, US and Canada only
+1 800 685 2999